

Installation Guide

Slide-on (plunger plate) Mounting Adapter for Masimo Radical Docking Station


The purpose of this guide is to:

1. Describe attachment of Mounting Adapter to docking station (pages 1 - 2).
2. Describe attachment of PRC™ (Post Rail Clamp) to Mounting Adapter (page 3).

Parts Reference

The following parts and hardware are included in this installation kit (hardware not shown):


Item #	Description	Qty
1	Mounting Adapter, Radical Docking Station	1
2	#6-32 x 1/2" Pan Head Machine Screw (PHMS)	4
3	#6-32 x 5/16" PHMS	4
4	#6 Flat Washer	4
5	1/4" Spacer (1/2" x .194" x 1/4"), Nylon	4

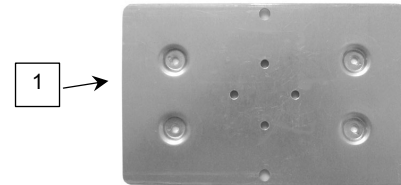

Tools Required: Phillips screwdriver (not provided).


Attachment of Mounting Adapter for Horizontal Mounting of Docking Station

1. Turn docking station upside down and place one (1) 1/4" Spacer over each mounting hole as shown below.
2. Fasten Mounting Adapter to bottom of docking station with four (4) #6-32 x 1/2" PHMS and #6 flat washers as shown below left. **CAUTION: Do not over tighten these screws.**

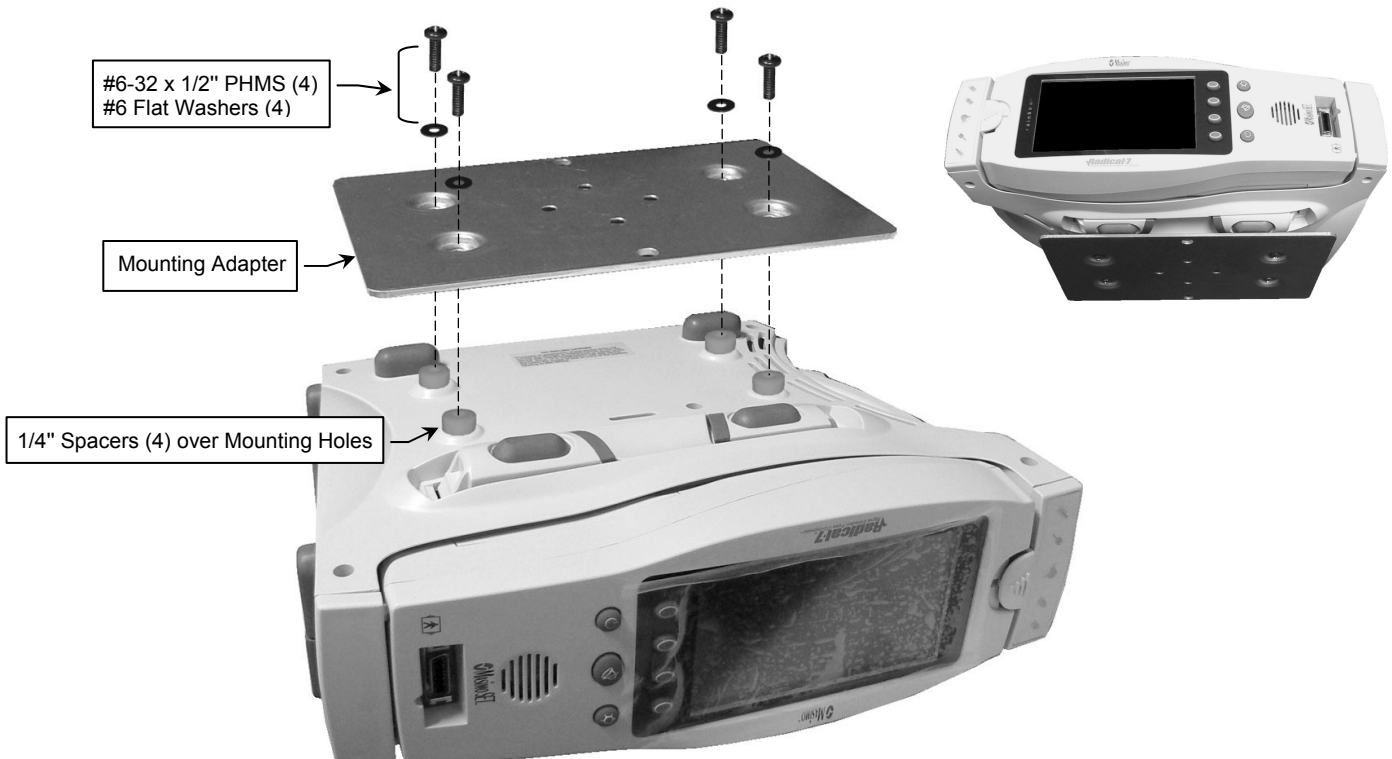

Attachment of Mounting Adapter for Vertical Mounting of Docking Station


1. Fasten Mounting Adapter to end of docking station with four (4) #6-32 x 5/16" PHMS and #6 flat washers as shown below left. **CAUTION: Do not over tighten these screws.**

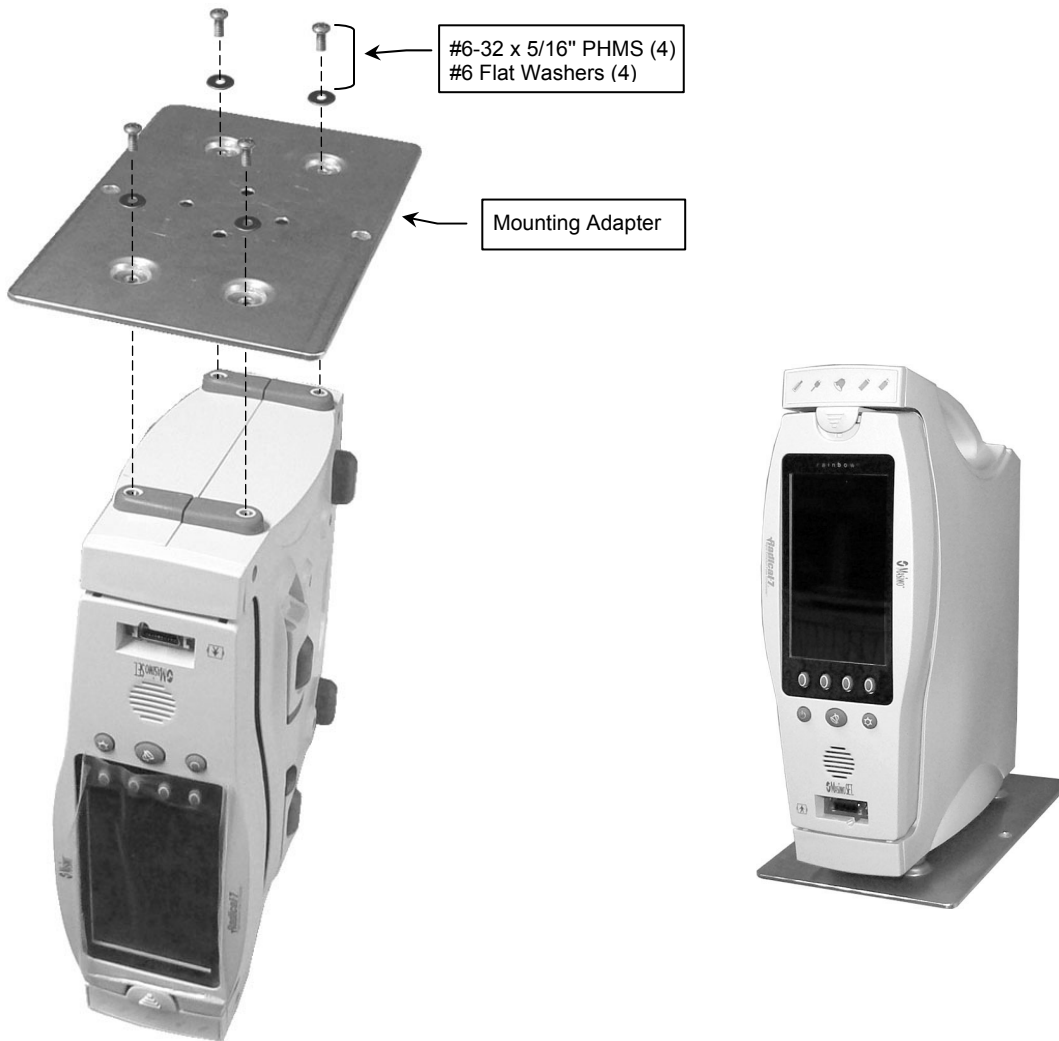

Mounting Docking Station on Post Using PRC™ (Post Rail Clamp)

Installation Note: PRC and referenced mounting screws are not included in this installation kit. Order GCX part number PRC-0001-01 to receive the PRC and #10-32 x 5/16" Flat Head Machine Screws.

1. Fasten Mounting Adapter to PRC as shown below with four (4) #10-32 x 5/16" FHMS (100°).

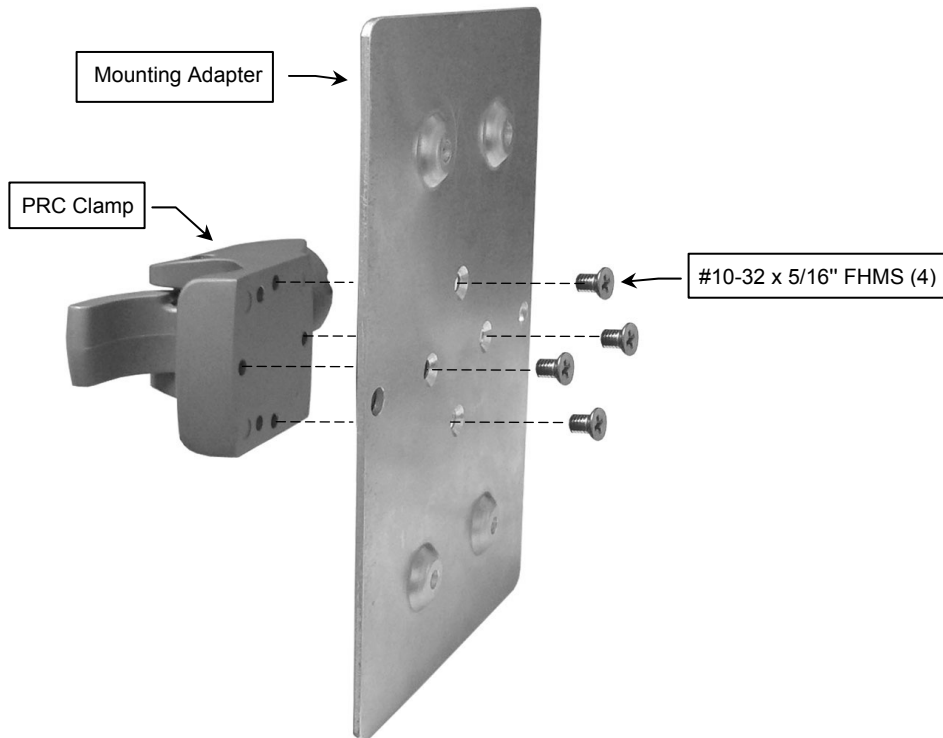

2. Fasten Mounting Adapter to docking station as described on page 1, steps 1 and 2 of this document.



Periodic Maintenance

Periodically inspect all fasteners associated with the mounting system. Tighten or adjust fasteners as necessary for optimal operation and safety.

Cleaning the Mounting Assembly

1. The mounting assembly may be cleaned with most mild, non-abrasive solutions commonly used in the hospital environment (e.g. diluted bleach, ammonia, or alcohol solutions).
2. The surface finish will be permanently damaged by strong chemicals and solvents such as acetone or trichloroethylene.
3. Do not use Steel wool or other abrasive material.
4. Damage caused by the use of unapproved substances or processes will not be warranted. It is recommended that you test any cleaning solution on a small area of the mounting assembly that is not visible to verify compatibility.
5. Never submerge or allow liquids to enter the mounting assembly. Wipe any cleaning agents off of the mounting assembly immediately, using a water-dampened cloth. Dry the mounting assembly thoroughly after cleaning.

CAUTION: GCX makes no claims regarding the efficacy of the listed chemicals or processes as a means for controlling infection. Consult your hospital's infection control officer or epidemiologist. To clean or sterilize mounted devices or accessory equipment, refer to the specific instructions delivered with those products.