

## Installation Guide

### Mounting Adapter for Tram-rac Module Housing


The purpose of this installation guide is to describe attachment of the Mounting Adapter to the Tram-rac housing.

#### Parts Reference

The following parts and hardware are provided for this procedure:

Item #	Description	Qty
1	Mounting Adapter	1
2	#6-32 x 5/16" Pan Head Machine Screw (PHMS)	4
3	#6 Flat Washer	4



Mounting Adapter

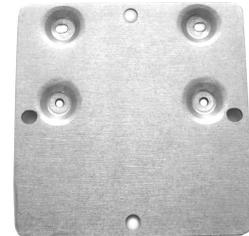

**Tools Required:** Phillips screwdriver.

#### Attaching Mounting Adapter to Tram-rac Housing

1. Turn Tram-rac housing upside down and locate four (4) threaded mounting holes shown below left. Fasten Mounting Adapter to housing with four (4) #6-32 x 5/16" PHMS and #6 flat washers (below right).

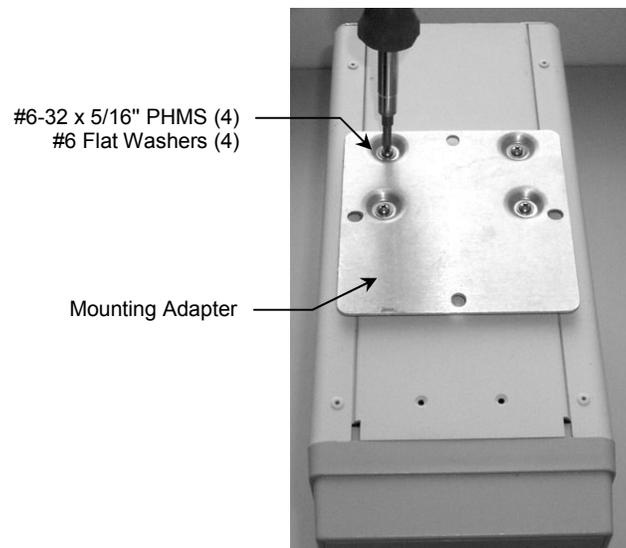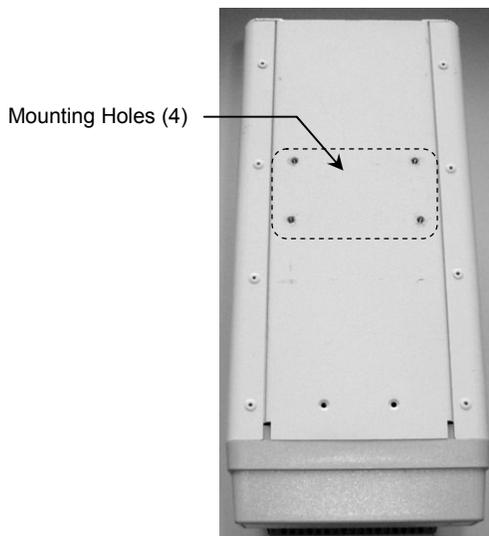

2. (Not shown) Slide Tram-rac onto Mounting Plate until Mounting Adapter makes contact with Plunger. Pull and hold Plunger while sliding Adapter over Plunger. Release Plunger and continue sliding Tram-rac until Plunger snaps into clearance hole in Mounting Adapter. Tighten Nylon screws on bottom of Mounting Plate to further secure Tram-rac.