

Installation Guide

Mindray T5/T8 Mounting Adapter with Hooks

The purpose of this guide is to:


- Describe attaching Mounting Adapter to T5/T8 Monitor.

Parts Reference

The following parts are included in this installation kit :

Item #	Description	Qty
1	T5/T8 Mounting Adapter	1
2	M4 x 20mm Pan Head Machine Screw (PHMS)	4

(Hardware not shown)

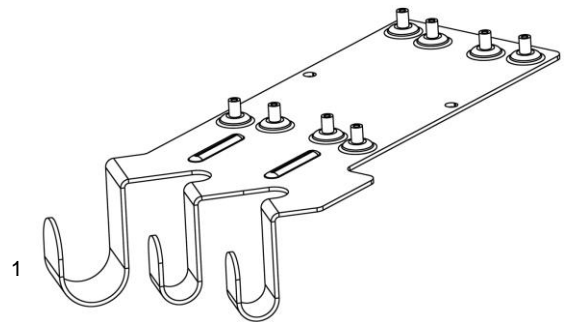


Tools Required:

- Phillips Screwdriver

Attaching T5/T8 Mounting Adapter to Monitor

1. Using a Phillips screwdriver, attach Mounting Adapter to Monitor with (4) M4 x 20mm PHMS as shown below.

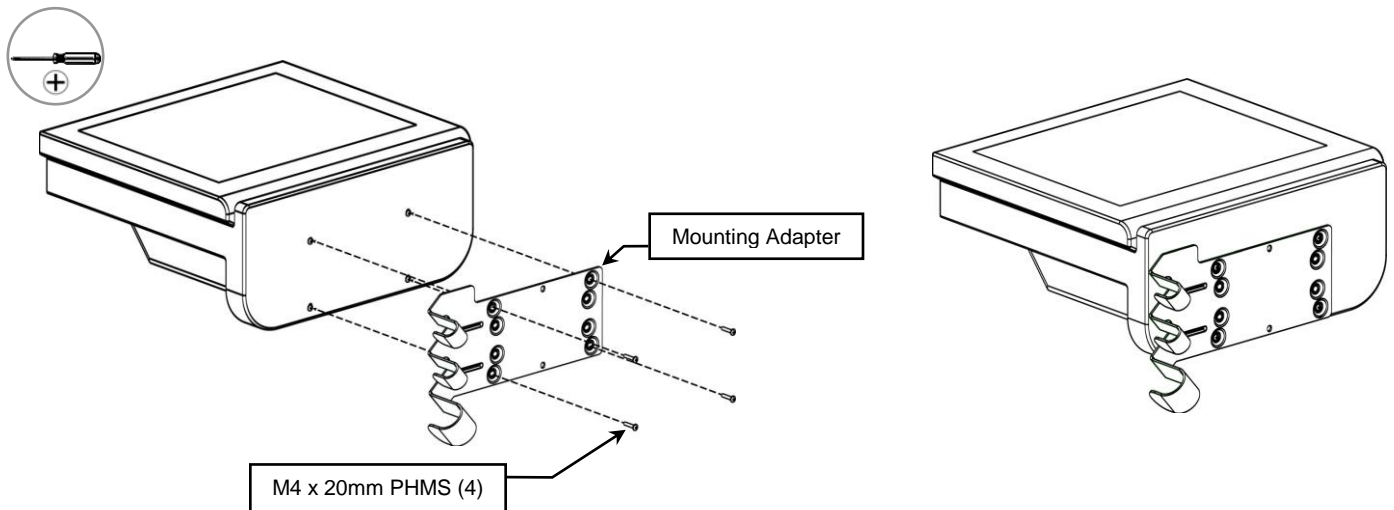

Periodic Maintenance

All fasteners associated with the mounting system should be inspected periodically and tightened as necessary.

Cleaning the Mounting Assembly

1. The mounting assembly may be cleaned with most mild, non-abrasive solutions commonly used in the hospital environment (e.g. diluted bleach, ammonia, or alcohol solutions).
2. The surface finish will be permanently damaged by strong chemicals and solvents such as acetone or trichloroethylene.
3. Steel wool or other abrasive material should never be used.
4. Damage caused by the use of unapproved substances or processes will not be warranted. It is recommended that you test any cleaning solution on a small area of the mounting assembly that is not visible to verify compatibility.
5. Never submerge or allow liquids to enter the mounting assembly. Wipe any cleaning agents off the mounting assembly immediately, using a water-dampened cloth. Dry thoroughly after cleaning.

CAUTION: GCX makes no claims regarding the efficacy of the listed chemicals or processes as a means for controlling infection. Consult your hospital's infection control officer or epidemiologist. To clean or sterilize mounted devices or accessory equipment, refer to the specific instructions delivered with those products.