

Installation Guide


Nihon Kohden BSM-3500 & 3700/PVM-2701 Mounting Adapter

The purpose of this guide is to describe the installation of the Mounting Adapter to the Monitor.

Parts Reference

The following parts and hardware are included in this installation kit (hardware not shown).

Item #	Description	Qty
1	Mounting Adapter	1
2	M3 x 8mm Pan Head Machine Screw (PHMS)	6

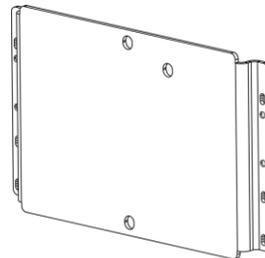


Tools Required

- Phillips screwdriver (not provided).

Attaching Mounting Adapter to PVM 2701

 **Installation Note:** Remove the (5) rubber feet attached to the bottom of the monitor prior to installing the mounting adapter.

1. Align the (4) Mounting holes on the Mounting Adapter with the (4) Threaded holes on monitor. Make sure to position Battery Access hole on the Mounting Adapter towards the back of the monitor.
2. Using a Phillips screwdriver, secure the Mounting Adapter to the Monitor with (4) M3 x 8mm PHMS.

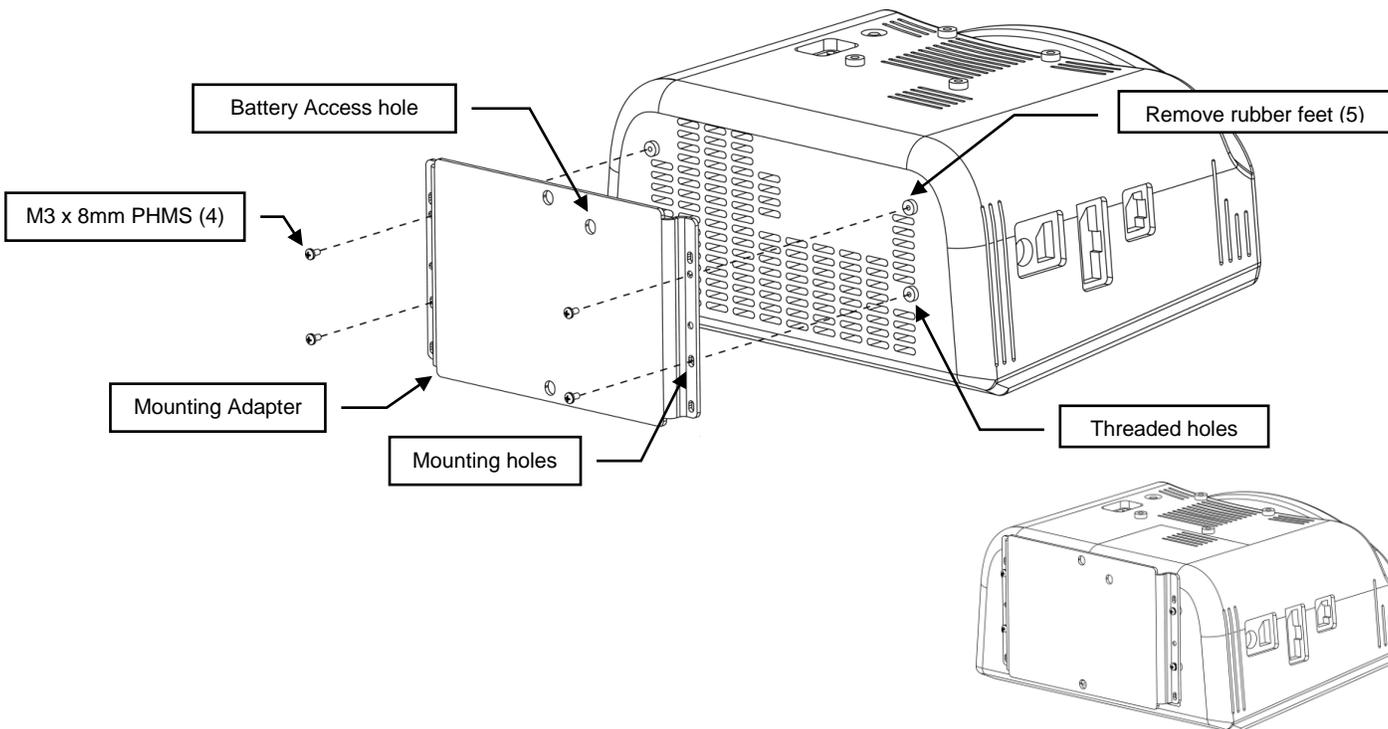


Attaching Mounting Adapter to 3000 Series Monitor



Installation Note: Remove the (5) rubber feet attached to the bottom of the monitor prior to installing the mounting adapter.

1. Align the (6) Mounting holes on the Mounting Adapter with the (6) Threaded holes on monitor. Make sure to position Battery Access hole on the Mounting Adapter towards the back of the monitor.
2. Using a Phillips screwdriver, secure the Mounting Adapter to the Monitor with (4) M3 x 8mm PHMS.

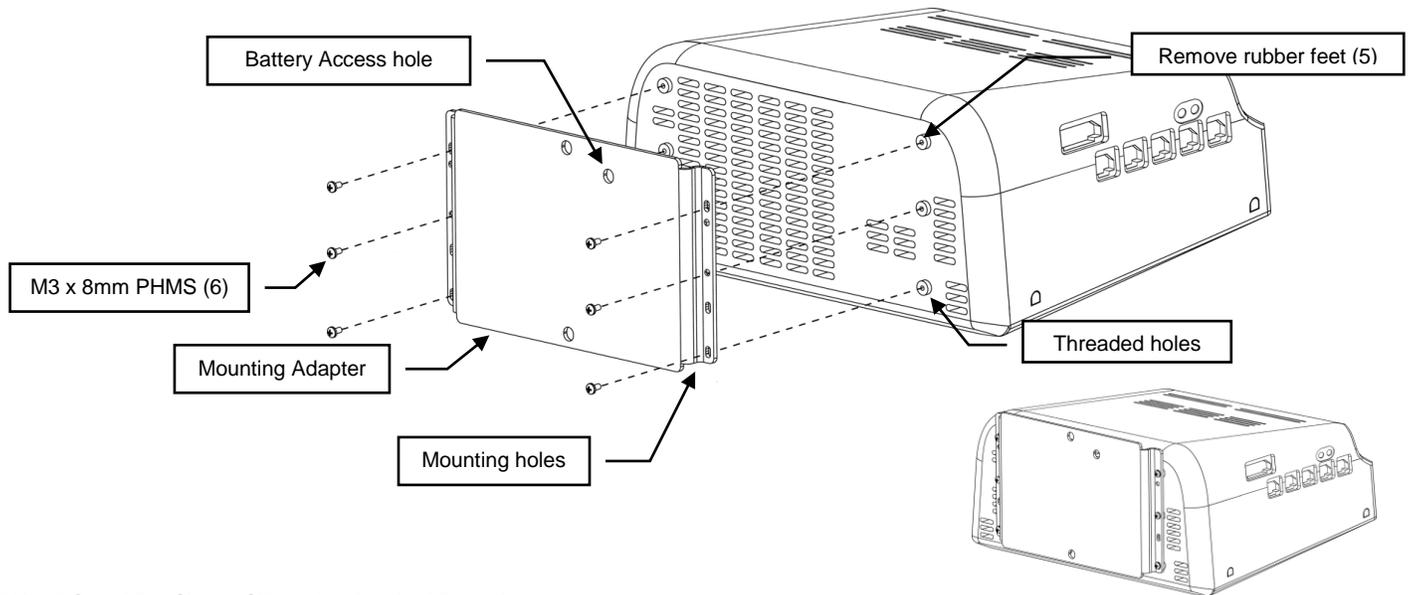

Attaching Monitor with Adapter to Mount

1. Pull Spring Plunger at the front of the Mounting Plate and slide Monitor laterally into the Mounting Plate until the Plunger snaps into the clearance hole at the front of the Mounting Adapter.
2. Tighten the Nylon Locking Screws located on bottom of the Mounting Plate.

