

Installation Guide
GCX® Overbed Table (OBT) Bin/Cupholder
(Refer to qualified personnel)

This installation guide covers the assembly of the OBT Bin/Cupholder to the OBT Tabletop.

Parts Reference

The following parts and hardware are included in this installation kit:


Item #	Description	Qty.
1	Bin/Cupholder	1

Tools Required

Phillips Screwdriver (Not Provided)


OBT Bin/ Cupholder Installation

1. Prepare the Bin/Cupholder to install onto the edge of the OBT Tabletop. Using a Phillips Screwdriver (not provided), loosen the two (2) screws on the bottom of the Bin/Cupholder without removing the screws.

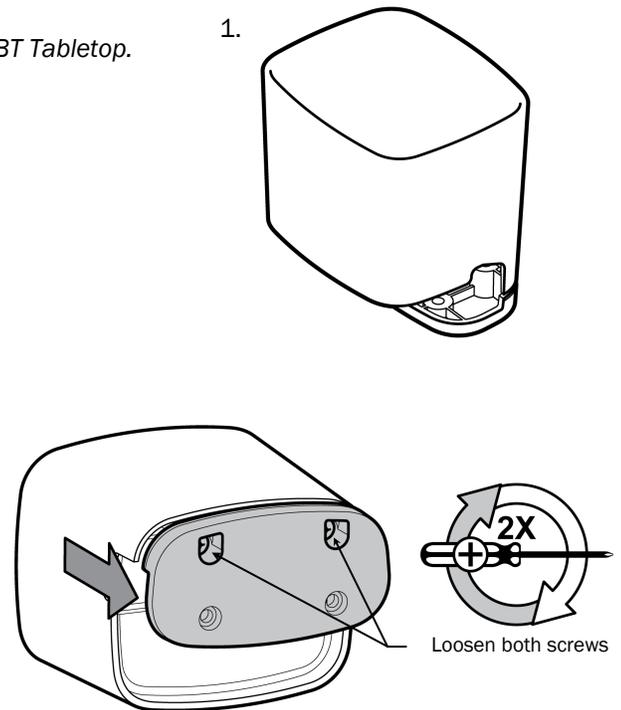


2. Fit the Bin/Cupholder over the edge of the OBT Tabletop.

Installation Note: The Bin/Cupholder can be placed along the sides, front, or rear of the Tabletop. Avoid placing on corners and blocking access to the drawer and the height adjustment paddles.

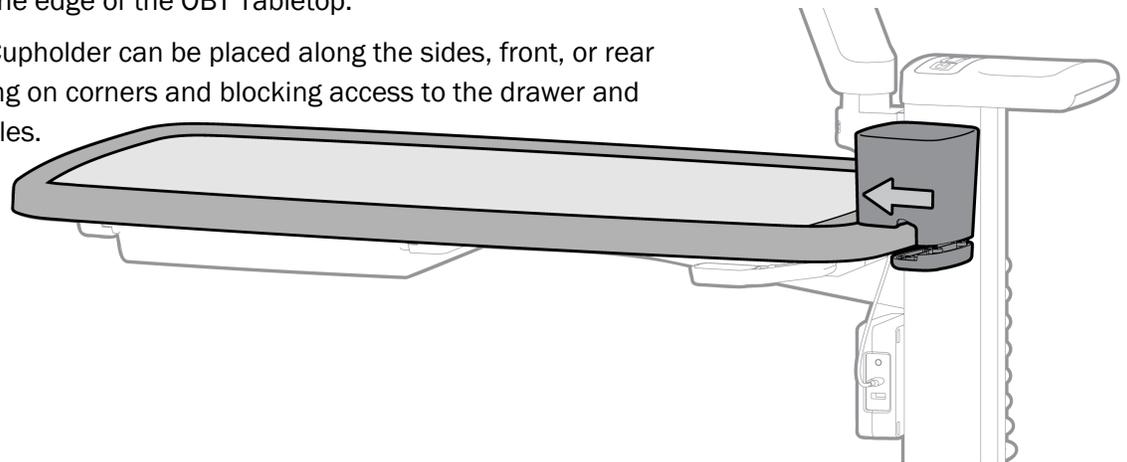

3. Once Bin/Cupholder is hooked over the edge of the Tabletop, using a Phillips Screwdriver, tighten both screws until Bin/Cupholder is secured.

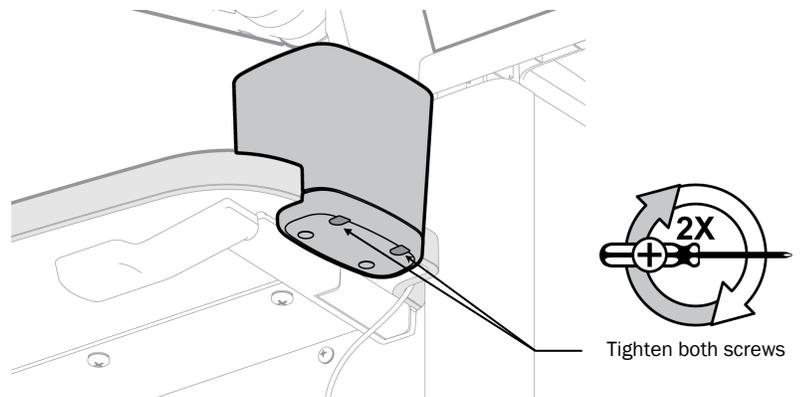

Routine Maintenance

Periodically inspect all fasteners associated with the mounting assembly. Tighten or adjust as necessary for optimal operation and safety.

Cleaning the Mounting Assembly

The mounting assembly may be cleaned with most mild, non-abrasive solutions commonly used in the hospital environment (e.g. diluted bleach, ammonia, or alcohol solutions).

The surface finish will be permanently damaged by strong chemicals and solvents such as acetone or trichloroethylene.

Steel wool or other abrasive material should never be used.

Damage caused by the use of unapproved substances or processes will not be warranted. We recommend testing of any cleaning solution on a small area of the mounting assembly that is not visible to verify compatibility.

Never submerge or allow liquids to enter the mounting assembly. Wipe any cleaning agents off the mounting assembly immediately using a water-dampened cloth. Dry the assembly thoroughly after cleaning.



CAUTION: GCX makes no claims regarding the efficacy of the listed chemicals or processes as a means for controlling infection. Consult your hospital's infection control officer or epidemiologist. To clean or sterilize mounted devices or accessory equipment, refer to the specific instructions delivered with those products.