

Installation Guide GCTX® Overbed Table (OBT) Drawer

(Refer to qualified personnel)

This installation guide covers the assembly of the OBT Drawer to the OBT Tabletop.


Parts Reference

The following parts and hardware are included in this installation kit:

Item #	Description	Qty.
1	OBT Drawer	1
2	OBT Drawer Rails	2
3	OBT Drawer Rail Latch	2
4	1/4-20 x 1" Flat Head Machine Screw (FHMS)	6


Tools Required

Phillips Screwdriver (Not Provided)

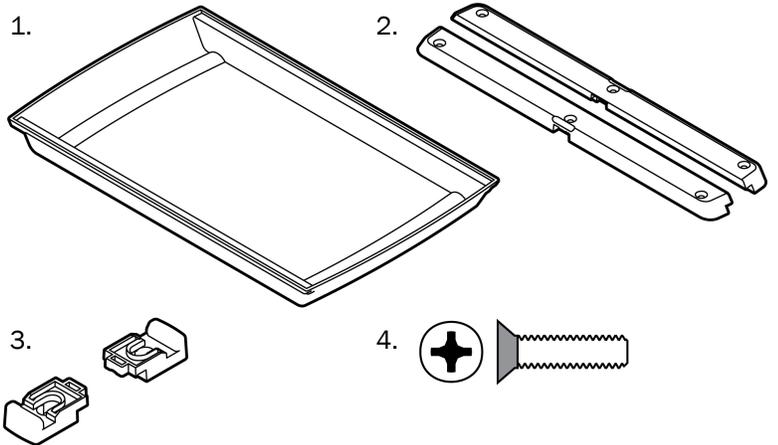


Assemble Drawer Rails and Latches

1. Remove all loose items from the OBT Tabletop and lay the OBT on its side to access the underside of the OBT Tabletop and locate the two (2) rows of three (3) threaded inserts.
2. Insert one (1) Drawer Latch into the center of each Drawer Rail.
3. Align the Drawer Rails with each set of threaded inserts (Drawer Latches pointing outward). Using a Phillips Screwdriver (not provided) install the Rails to the OBT Tabletop using three (3) 1/4-20 x 1" FHMS.

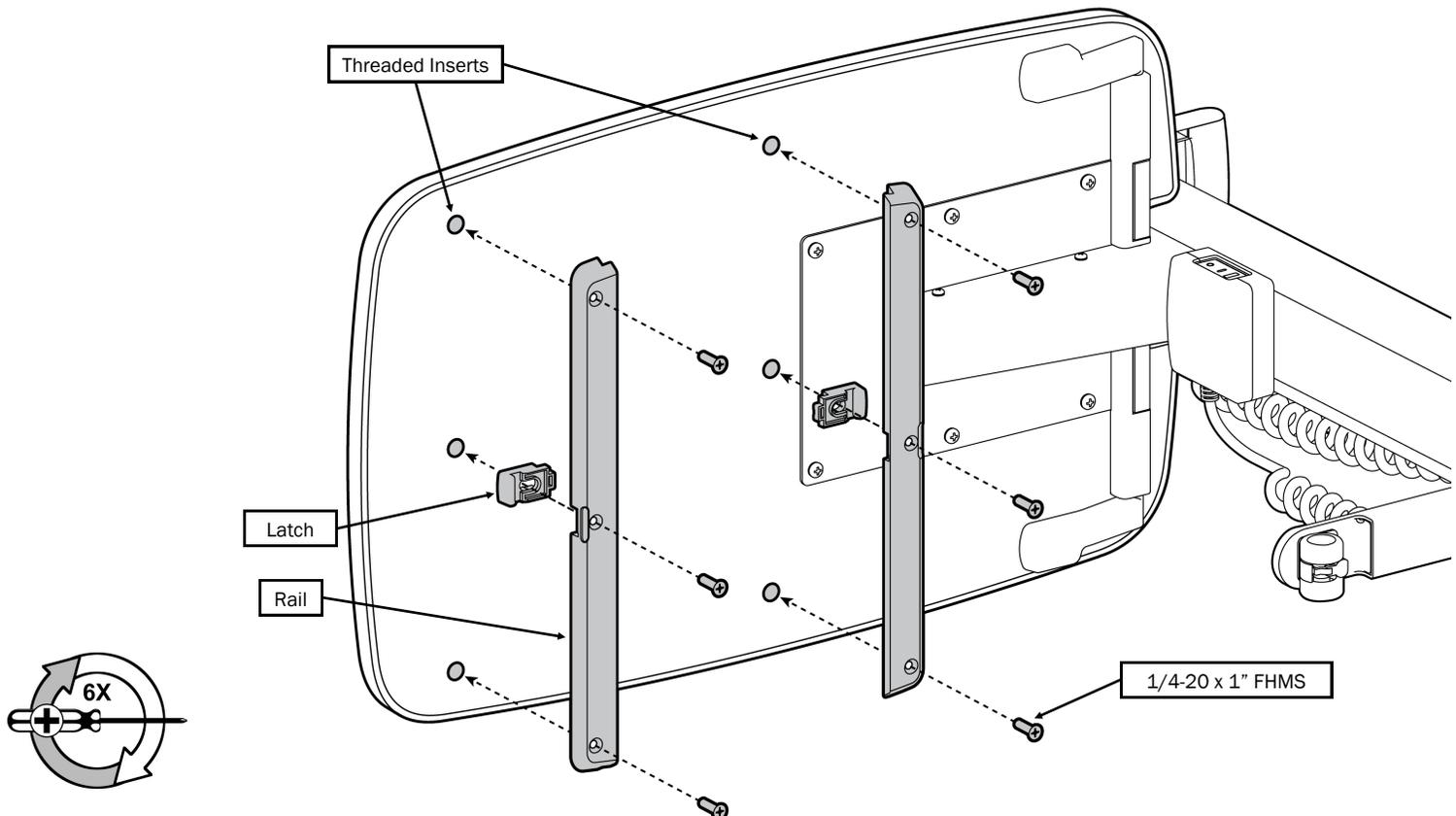


Installing the OBT Drawer

1. Pull each latch outward until they click.
2. Slide the OBT Drawer into the Rails.
3. Close the Drawer and press each Latch inward until they click. This will secure the Drawer within the Rails.
4. Lift the OBT back onto its casters.

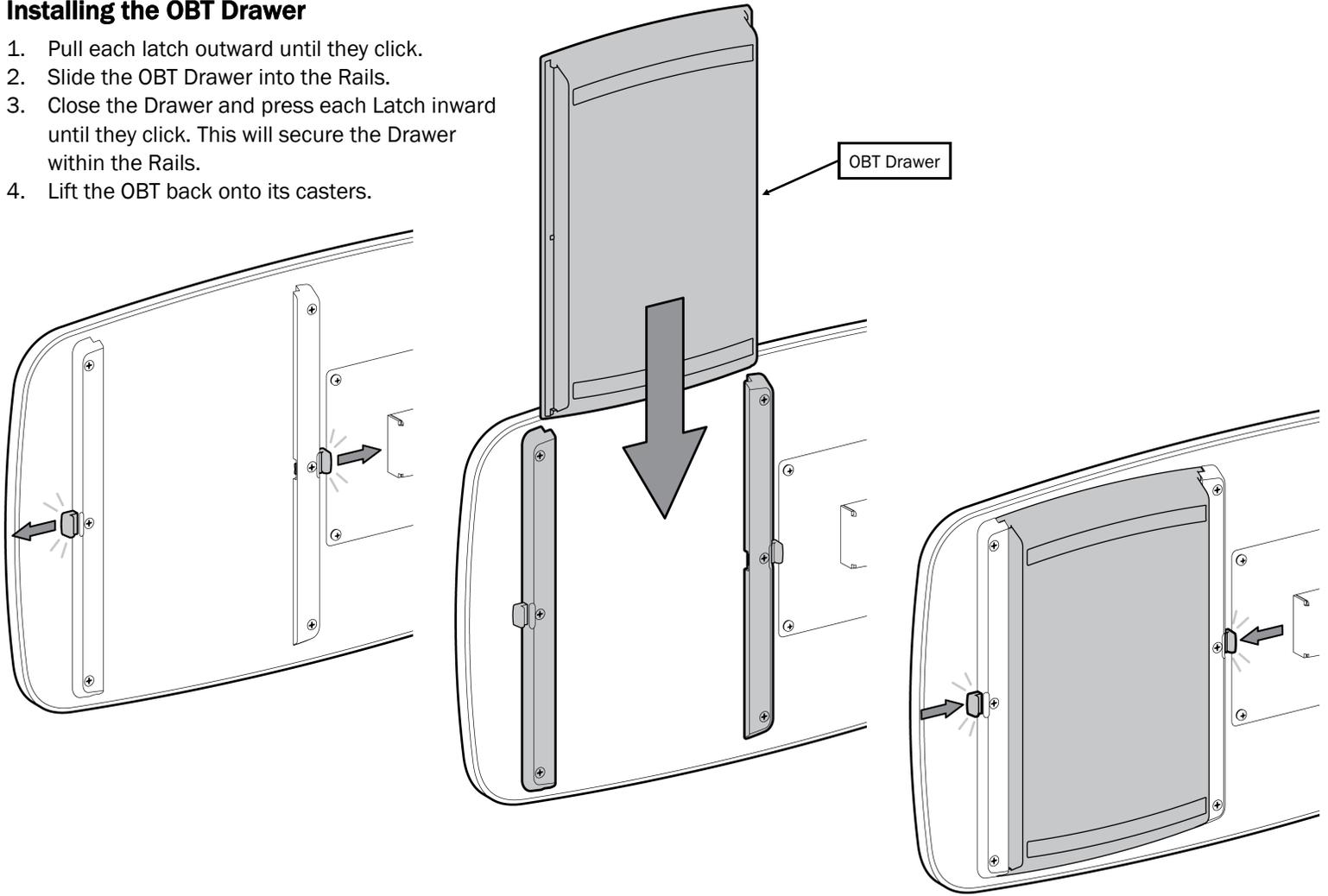

OBT Drawer Operation


The OBT Drawer is designed to be accessed from either side of the OBT Tabletop.

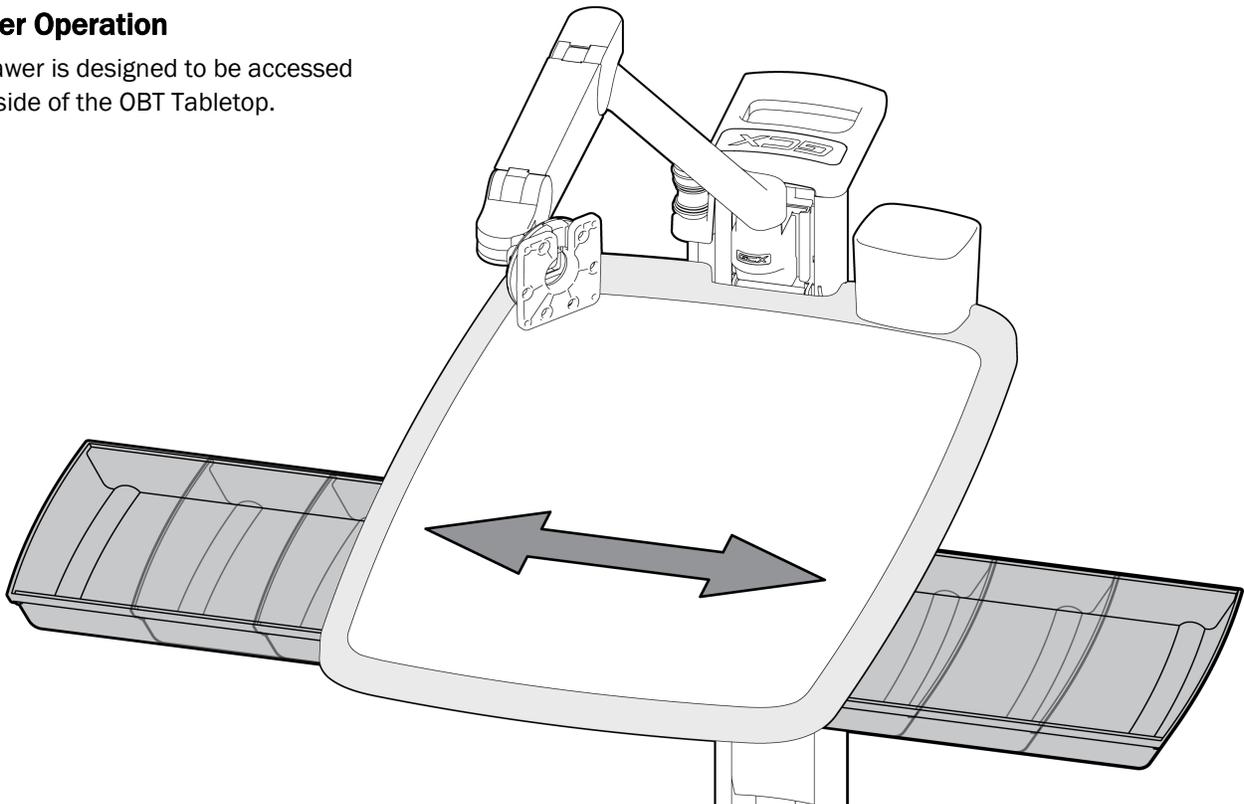


OBT Drawer Removal and Installation

The OBT Drawer can be removed for cleaning purposes.


1. To remove the Drawer, locate two latches (one on each side of the drawer) and pull each latch outward until they click.

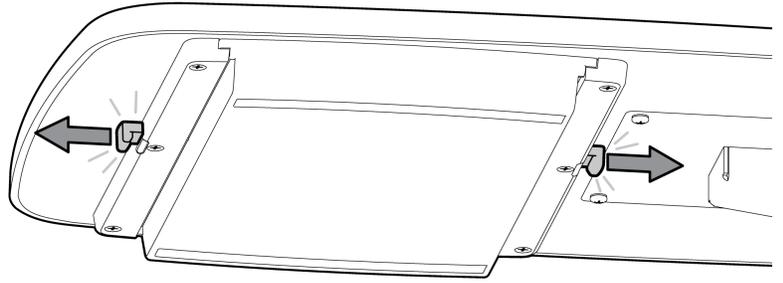


2. Once the Latches are unlocked, simply slide the Drawer completely out of the rails.

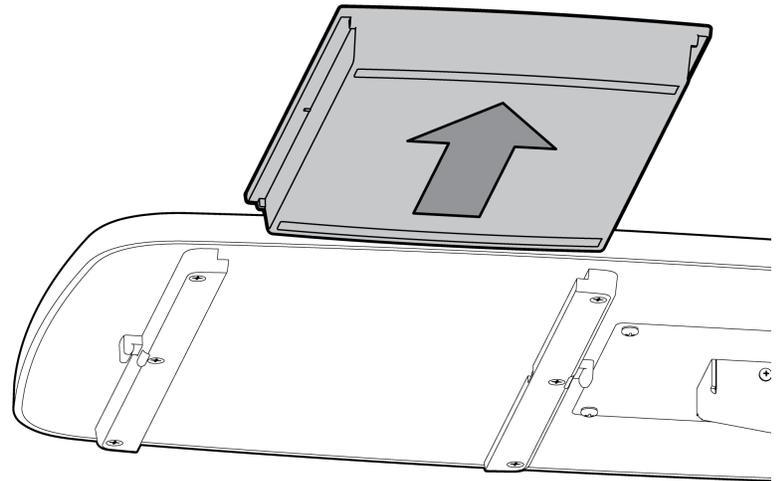

1. To reinstall the drawer, slide the Drawer back into the rails and close the drawer.

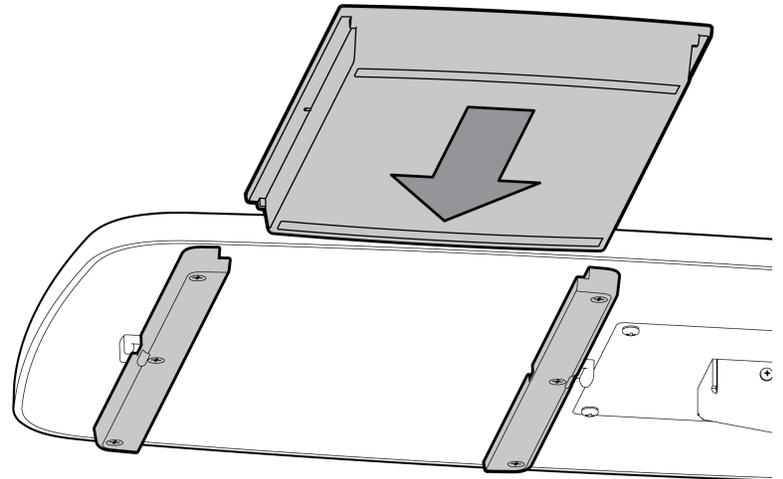

2. Once Drawer is in place, press the Latches inward until they click. The drawer is now secured.

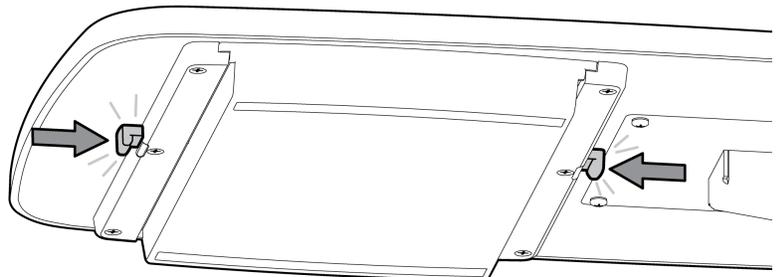

Routine Maintenance

Periodically inspect all fasteners associated with the mounting assembly. Tighten or adjust as necessary for optimal operation and safety.

Cleaning the Mounting Assembly

The mounting assembly may be cleaned with most mild, non-abrasive solutions commonly used in the hospital environment (e.g. diluted bleach, ammonia, or alcohol solutions).

The surface finish will be permanently damaged by strong chemicals and solvents such as acetone or trichloroethylene. Steel wool or other abrasive material should never be used.

Damage caused by the use of unapproved substances or processes will not be warranted. We recommend testing of any cleaning solution on a small area of the mounting assembly that is not visible to verify compatibility.

Never submerge or allow liquids to enter the mounting assembly. Wipe any cleaning agents off the mounting assembly immediately using a water-dampened cloth. Dry the assembly thoroughly after cleaning.



CAUTION: GCX makes no claims regarding the efficacy of the listed chemicals or processes as a means for controlling infection. Consult your hospital's infection control officer or epidemiologist. To clean or sterilize mounted devices or accessory equipment, refer to the specific instructions delivered with those products.