

GCX Mounting Assembly Installation Guide

Philips M3 Recorder – Wall Channel and 10mm Rail Mounts

The purpose of this guide is to:

1. Describe attachment of bracket to recorder, attachment of rail clamp to bracket, and mounting of recorder on wall and 10mm rail.

Parts Reference

The following parts and hardware are included with this installation kit (see photo for parts, hardware not shown):

Item #	Description	Qty
1	Mounting Bracket Assembly (Channel Slide Attached)	1
2	Rail Clamp	1
3	#4-40 x 5/16" Pan Head Machine Screw (PHMS)	4
4	#8-32 x 5/16" PHMS	4
5	3/16" Hex Key – for Rail Clamp Applications	1




Tools Required: Phillips screwdriver, 3/16" hex key (provided).

Installation Note: The Mounting Bracket is shipped from the factory with Slide attached for wall mounting. If your application calls for rail mounting, first attach the Mounting Bracket to the Recorder, then follow "Attachment of Rail Clamp to Mounting Bracket" procedure on page 2.

Attaching the Mounting Bracket to the Recorder


1. Turn Recorder upside down and remove four (4) existing mounting screws (or feet if attached) from the bottom housing (not shown).
2. The mounting side of the Bracket may be positioned on either the left or right side of the Recorder depending on application requirements. Align the mounting holes in the Bracket with the threaded holes in the bottom of the Recorder (Pic 1) and attach the Mounting Bracket to the Recorder using four (4) #4-40 x 5/16" PHMS (Pic 2).

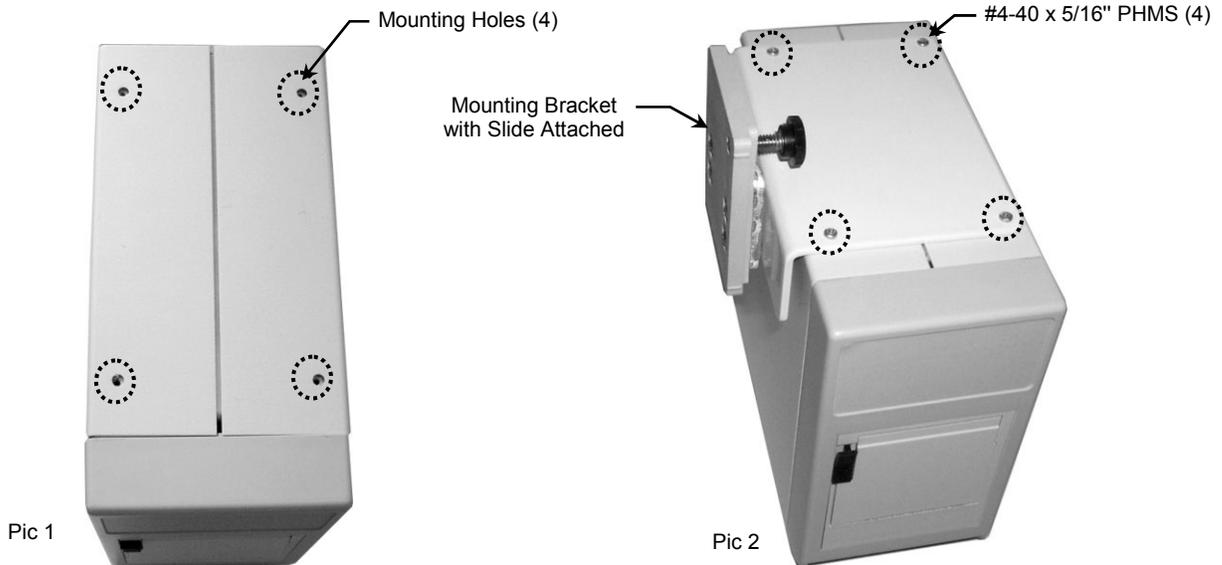

Attachment of Rail Clamp to Mounting Bracket

Installation Note: It is not necessary to remove the Mounting Bracket in order to attach the Rail Clamp.


1. Remove the Slide from the Mounting Bracket by unscrewing the four (4) #8-32 x 5/8" PHMS (not shown).
2. Attach the Rail Clamp to the Mounting Bracket using four (4) #8-32 x 5/16" PHMS (Pic 3). Proceed to "Rail Mounting (10mm Rail)" below.

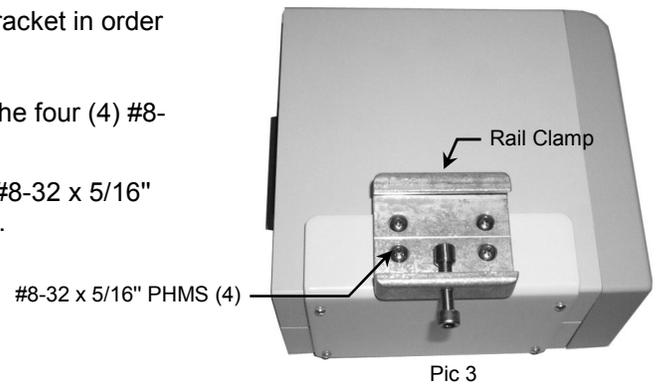

Mounting the Recorder


Wall Mounting

1. Insert the Slide in the top of the wall channel (Pic 4) and slide the Recorder to the desired mounting position. Tighten the Clamping Knob to secure the Recorder in position (Pic 5).

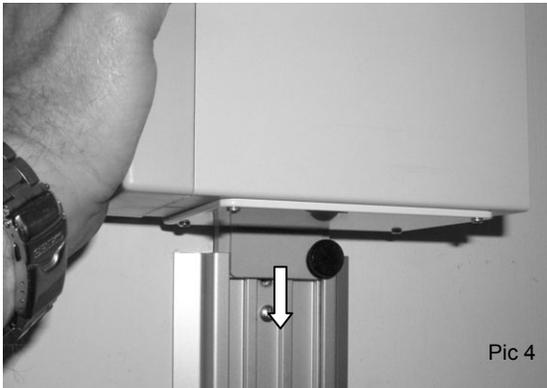

Rail Mounting (10mm Rail)

1. Hook the top of the Rail Clamp over the top of the 10mm rail (Pic 6) and tighten the Clamping Screw using the 3/16" hex key.

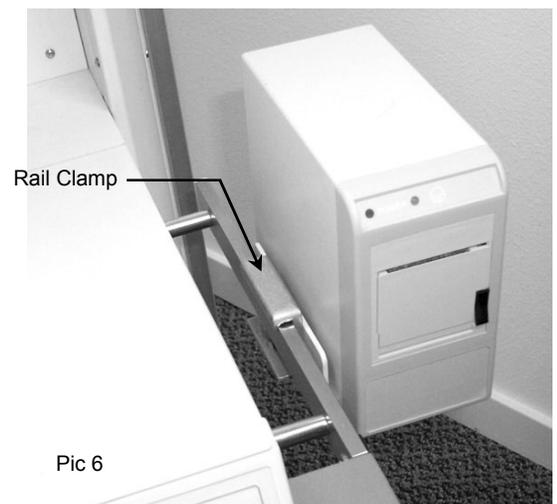

Cleaning the Mounting Assembly

1. The mounting assembly may be cleaned with most mild, non-abrasive solutions commonly used in the hospital environment (e.g., diluted bleach, ammonia, or alcohol solutions).
2. The surface finish will be permanently damaged by strong chemicals and solvents such as acetone and trichloroethylene.
3. Steel wool or other abrasive material should *never* be used.
4. Damage caused by the use of unapproved substances or processes will not be covered under warranty. It is recommended that you test any cleaning solution on a small area of the arm that is not visible, to verify compatibility.

CAUTION: GCX makes no claims regarding the efficacy of the listed chemicals or processes as a means for controlling infection. Consult your hospital's infection control officer or epidemiologist. To clean or sterilize mounted devices or accessory equipment, refer to the specific instructions delivered with those products.