

GCX Mounting Assembly Installation Guide

Wall Mount M-Series – PolyQuip Post Assemblies – 1" Diameter Up Post or Down Post

The purpose of this guide is to describe the attachment of the PolyQuip Post to the top or bottom of the M-Series Arm.

Parts Reference


The following parts and hardware are provided for this procedure:

Item #	Description	Qty
1	PolyQuip Post	1
2	#10-32 x 9/16" Socket Head Cap Screw (SHCS)	3
3	1/4-20 x 1/2" Button Head Socket Cap Screw (BHSCS)	1
4	1/4 x 1.375" OD Fender Washer	1
5	5/32" Hex Key	1
6	Lexan Cover, 2" Diameter	1

Tools Required: 5/32" hex key (provided).


Attaching the PolyQuip Post to the Top of the M-Series Arm

1. Using the 5/32" hex key provided, fasten PolyQuip Post to upper swivel fitting with three (3) #10-32 x 9/16" SHCS.

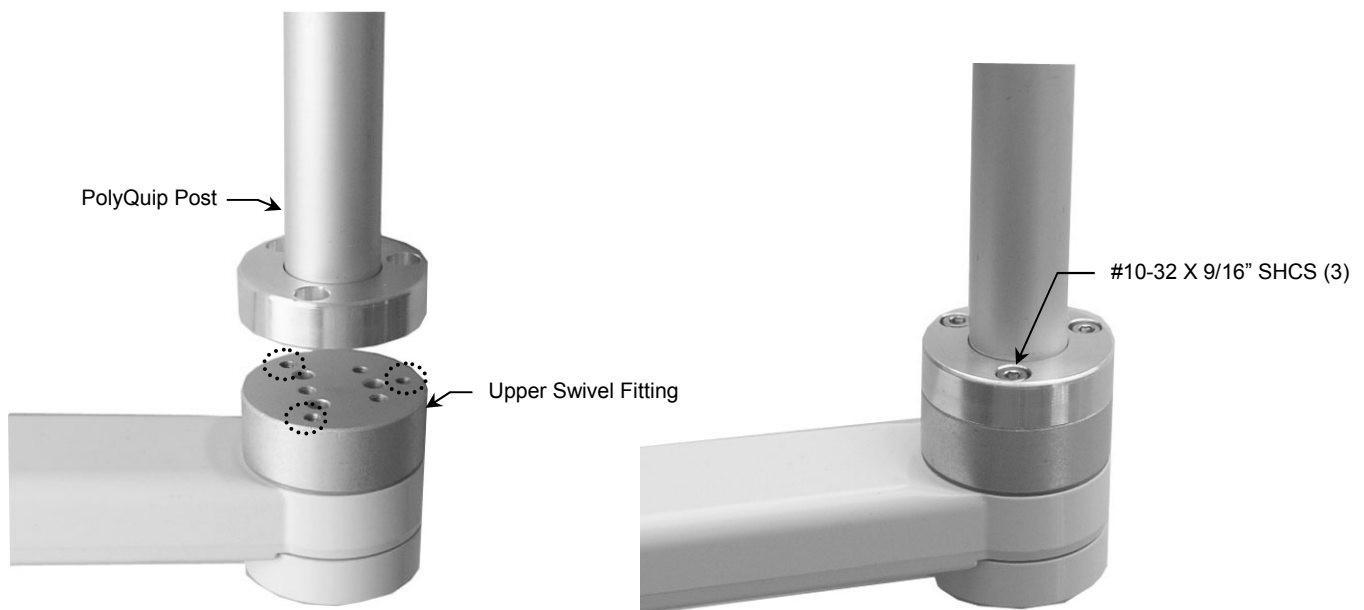

Attaching the 1" Diameter Post to the Bottom of the M-Series Arm

1. Remove plastic bolt cap from bottom of Arm. Using the 5/32" hex key provided, fasten PolyQuip Post to lower swivel fitting with three (3) #10-32 x 9/16" SHCS.

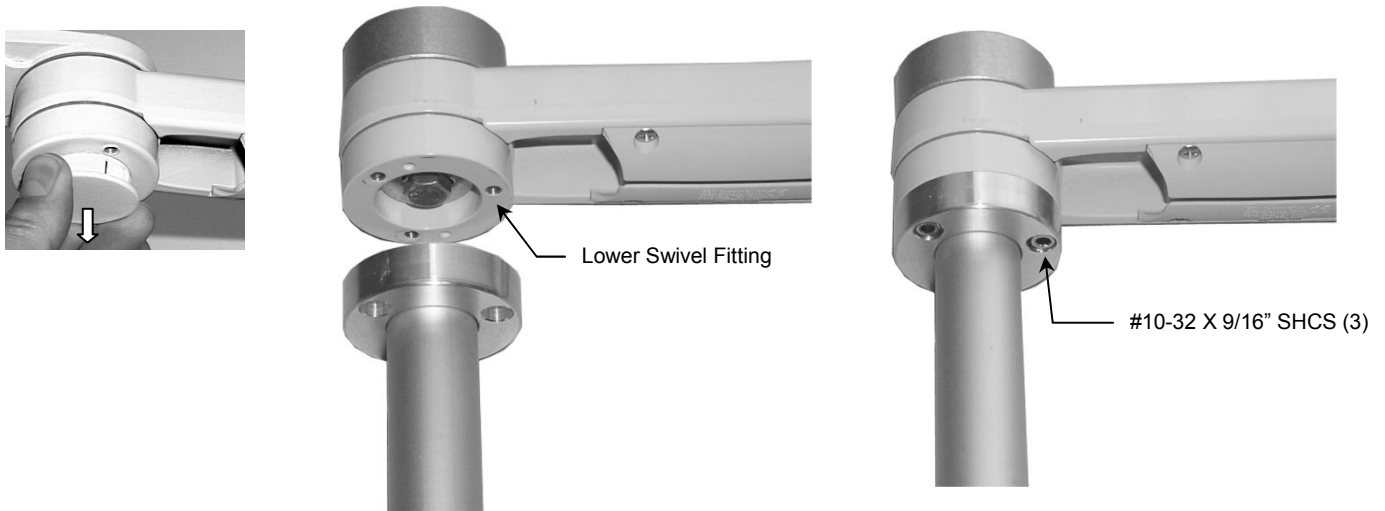

2. Remove black plastic post cap from end of Down Post (below left). Using the 5/32" hex key, fasten the 1/4 x 1.375" washer to bottom of Post with one (1) 1/4-20 x 1/2" BHSCS (below center). **Installation Note:** An adhesive-backed Lexan Cover (below right) is provided to cover the mounting holes in the upper swivel head if desired.

