


Installation Guide

PRC Post/Rail Clamp for Tablet with Ball Joint, 75/100mm VESA for .75"-1.5" / 19-38mm Post / 10mm X 25mm Rail

The Purpose of these instructions is to:


1. Describe attachment of Tablet Mount for PRC

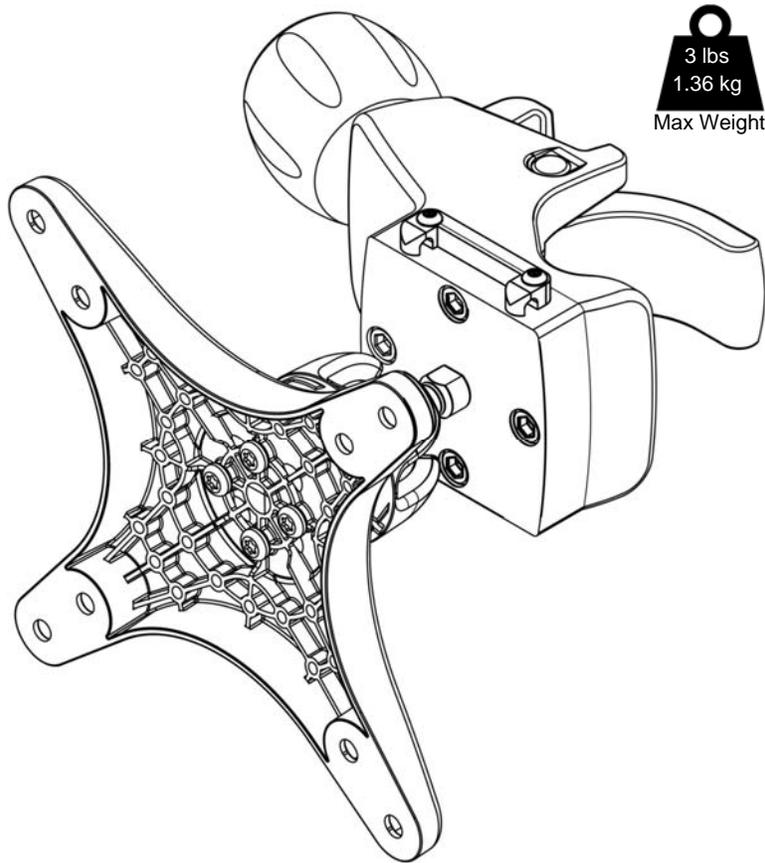

Parts Reference


The following parts and hardware are included in this installation kit:

Item #	Description	Qty.
1	Post/Rail Clamp with VESA Mount	1
2	Hardware kit (Refer to DU-FLP-0002-97)	1
3	M4 Nylon Locknut (Not Shown)	4

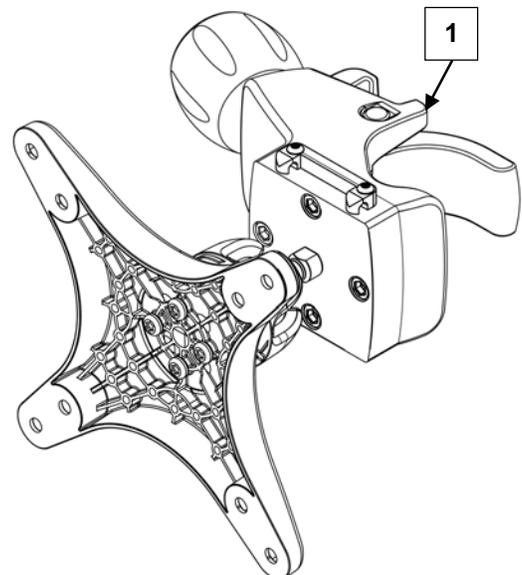






Tools Required:

- 5/64" Hex Wrench (Provided)
- Philips Screwdriver (Not provided)
- 9/32" (7mm) Socket Drive (Not provided)

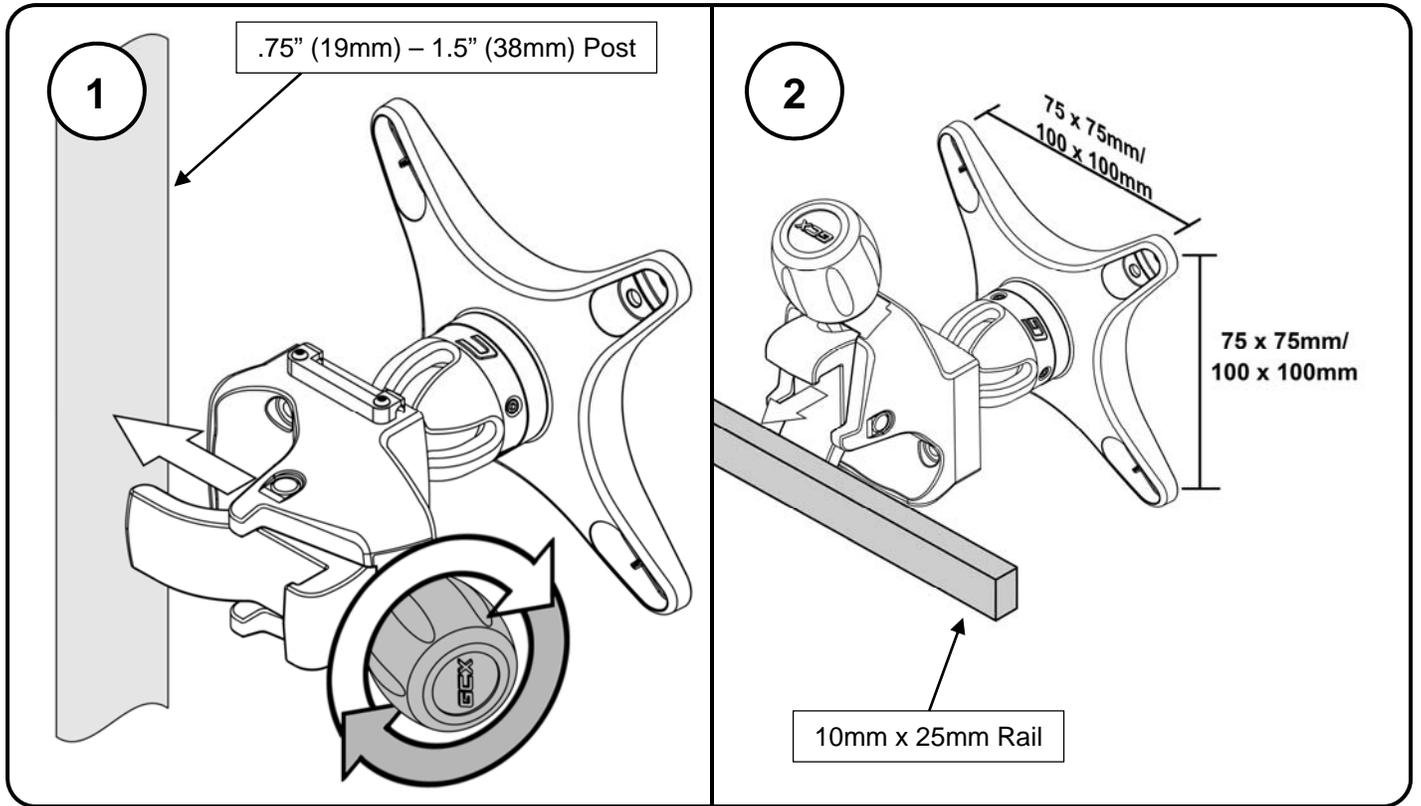


Steps 3 & 4 are dependent on Device and Enclosure

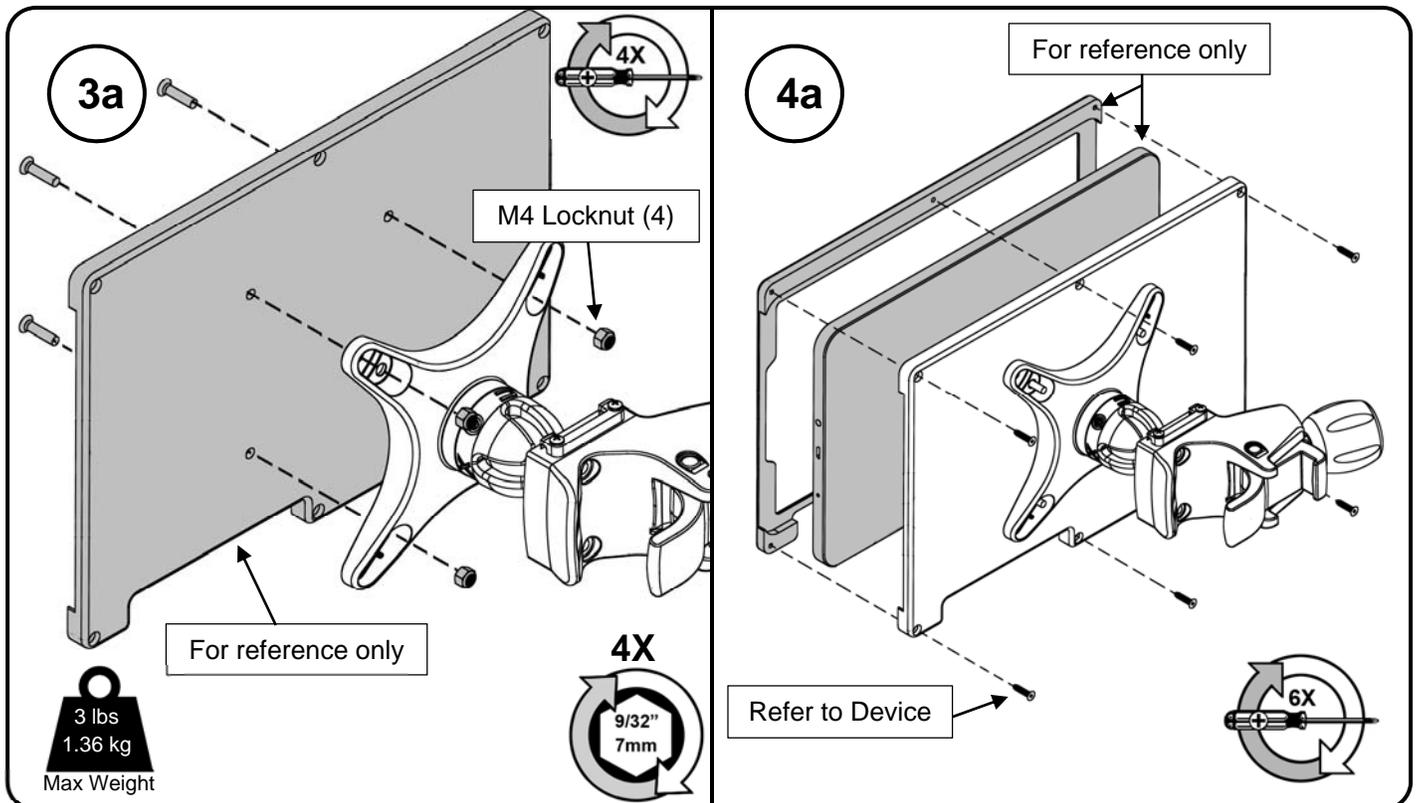


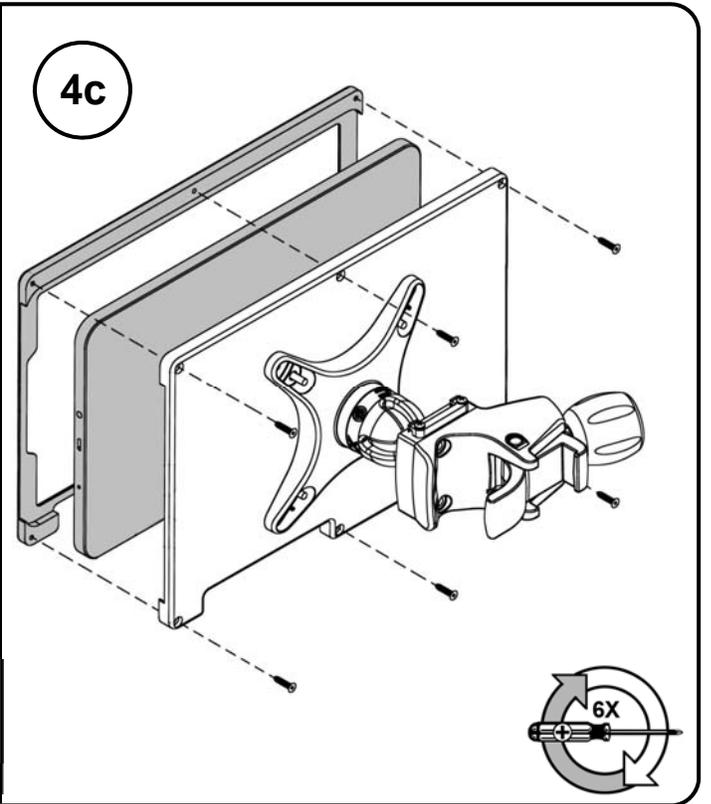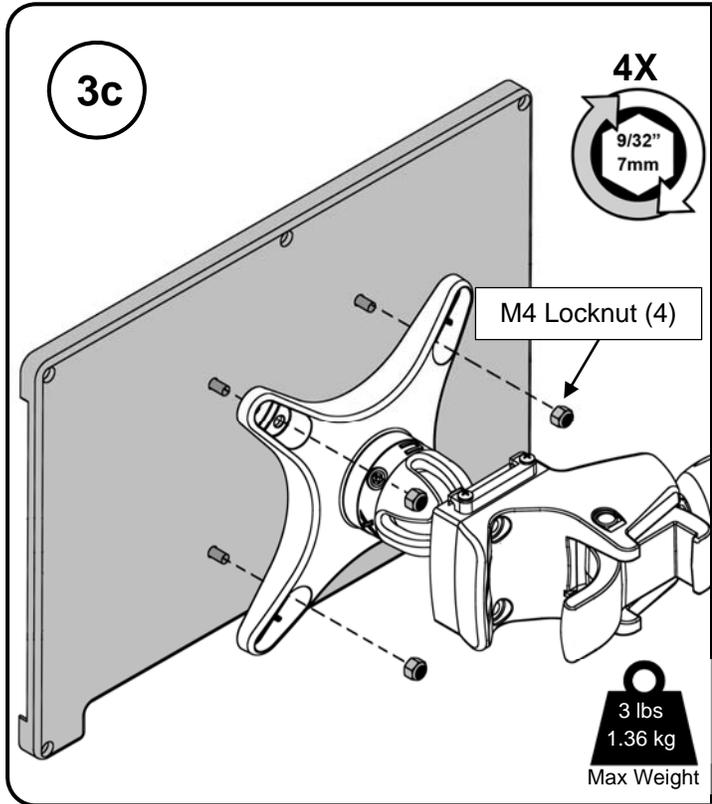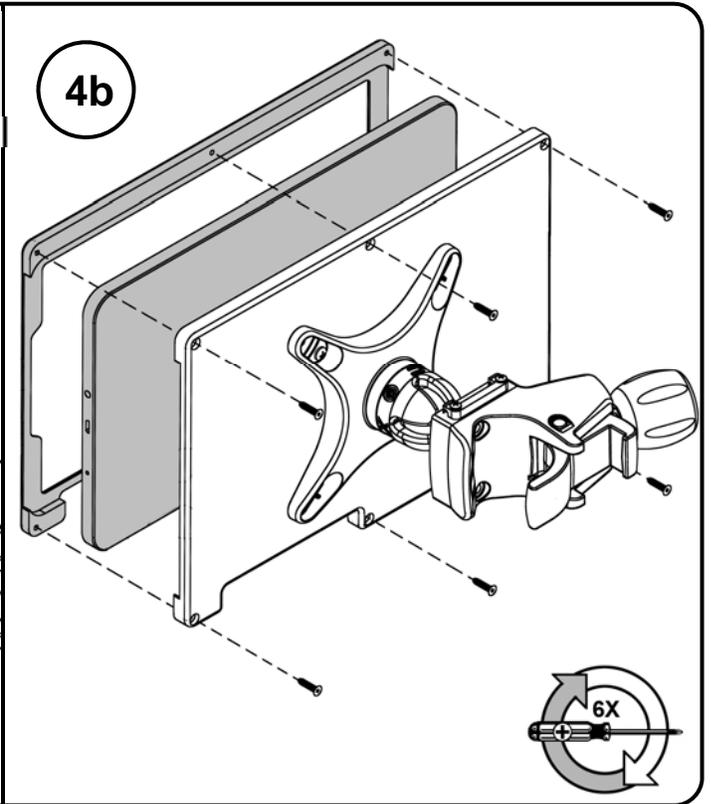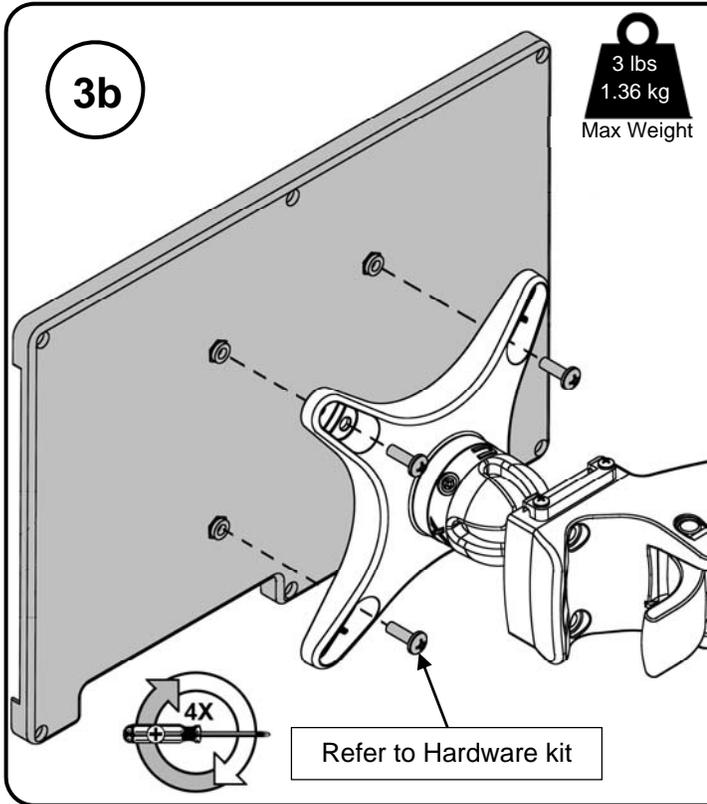

Swivel Tension

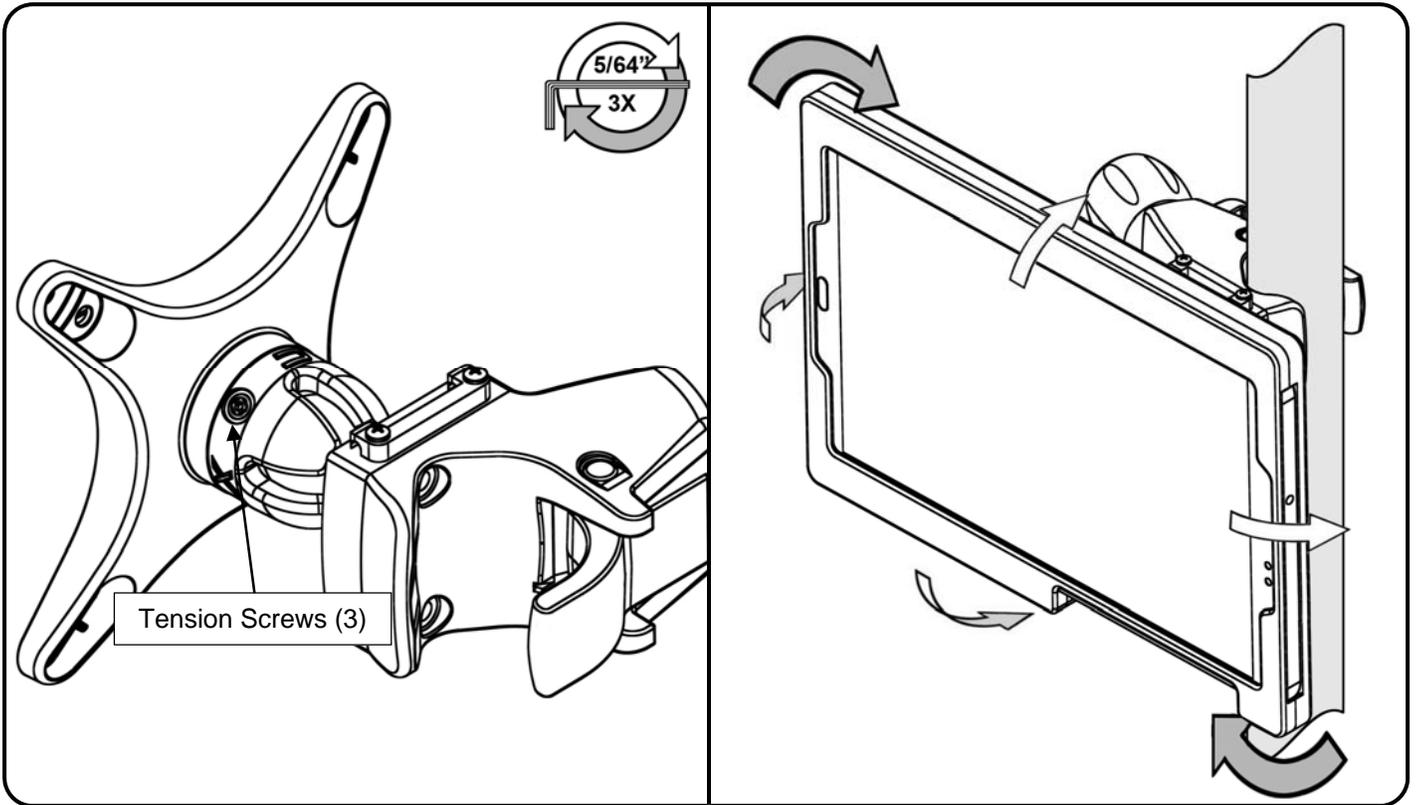

Cable Management

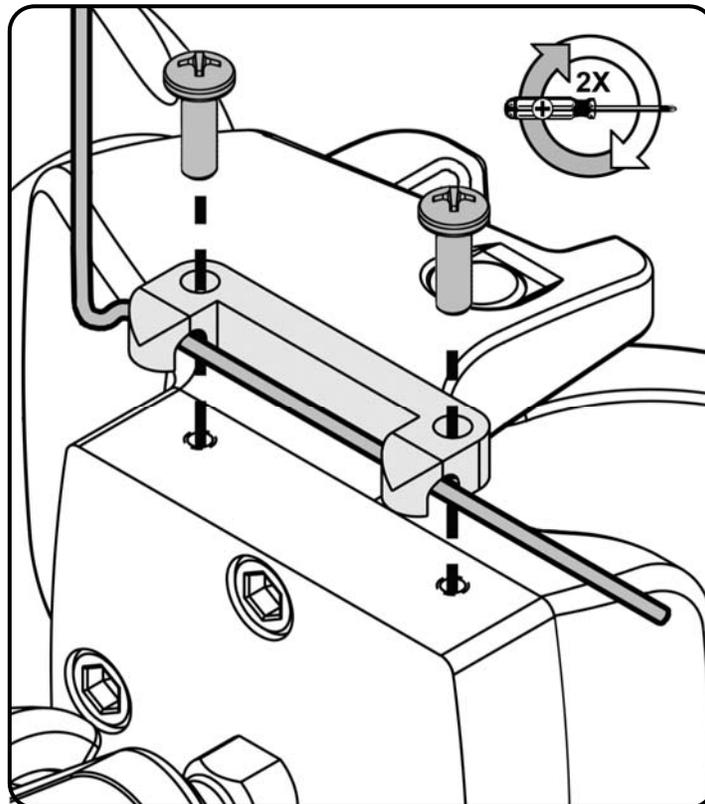

Routine Maintenance

Periodically inspect all fasteners associated with the mounting assembly. Tighten or Adjust as necessary for optimal operation and safety.

Cleaning the Mounting Assembly

The mounting assembly may be cleaned with most mild, non-abrasive solutions commonly used in the hospital environment (e.g. diluted bleach, ammonia, or alcohol solutions).

The surface finish will be permanently damaged by strong chemicals and solvents such as acetone or trichloroethylene.

Steel wool or other abrasive material should never be used.

Damage caused by the use of unapproved substances or processes will not be warranted. We recommend testing of any cleaning solution on a small area of the mounting assembly that is not visible to verify compatibility.

Never submerge or allow liquids to enter the mounting assembly. Wipe any cleaning agents off the mounting assembly immediately using a water-dampened cloth. Dry the assembly thoroughly after cleaning.

 **CAUTION:** GCX makes no claims regarding the efficacy of the listed chemicals or processes as a means for controlling infection. Consult your hospital's infection control officer or epidemiologist. To clean or sterilize mounted devices or accessory equipment, refer to the specific instructions delivered with those products.