


## Installation Guide

### 10.6" x 15.25" x 13" / 26.9cm x 38.7cm x 33cm Utility Basket for GCX Channel, VHRC/RC Rear Accessory Track or 2"/5.1cm Roll Stand Post

(Refer to qualified personnel)

**Purpose:** Describe the installation of Utility Basket



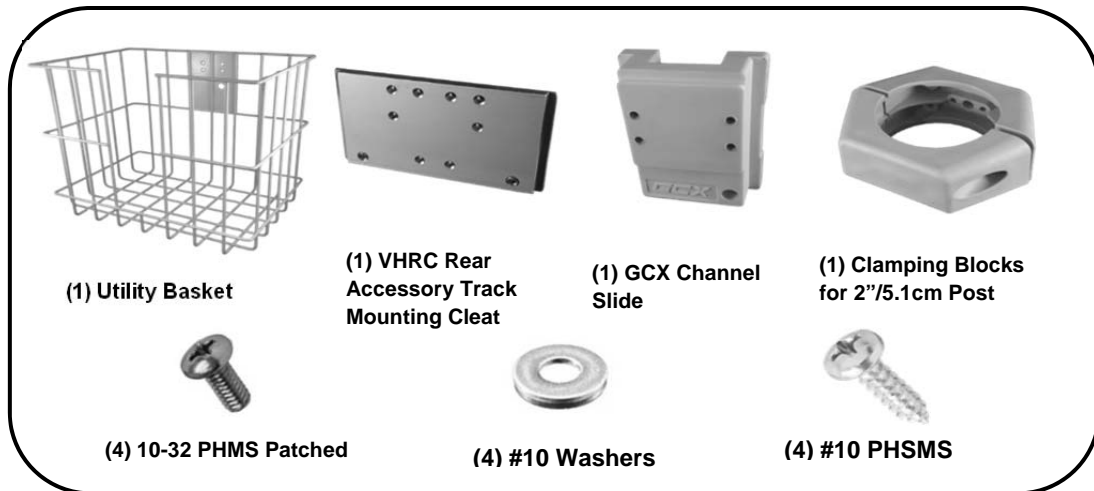


**WARNING:** When adding this basket to a mobile product, consider the tip stability of the finished system. Contact GCX for support or system testing.



#### Tools Required:

1/8" Hex Wrench (Provided)  
Phillips Screwdriver (Not provided)


# Variable Height Roll Cart Configuration Installation

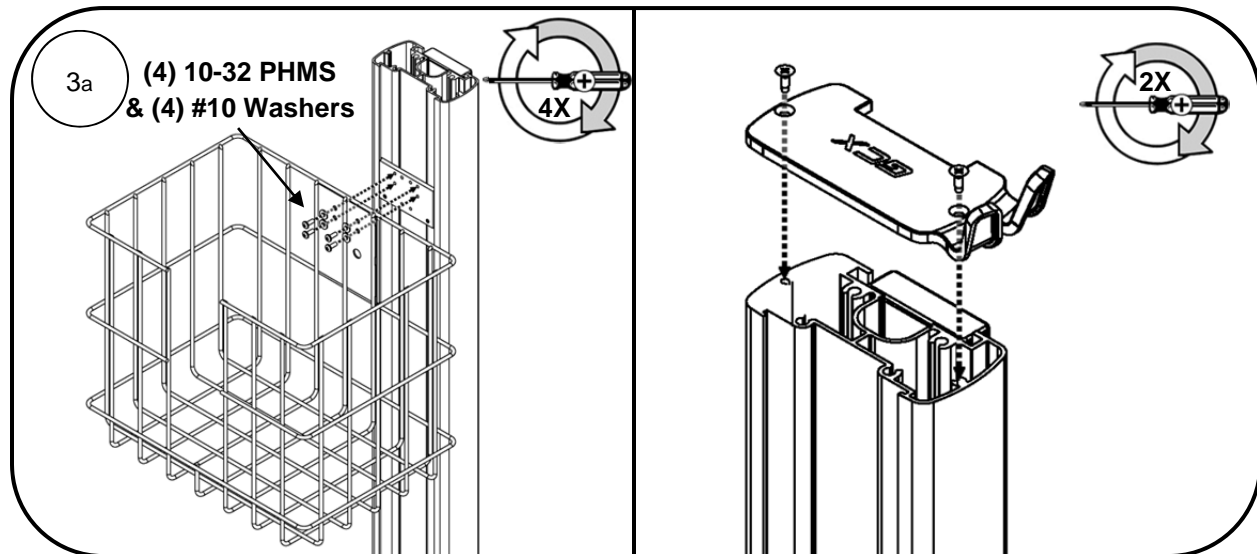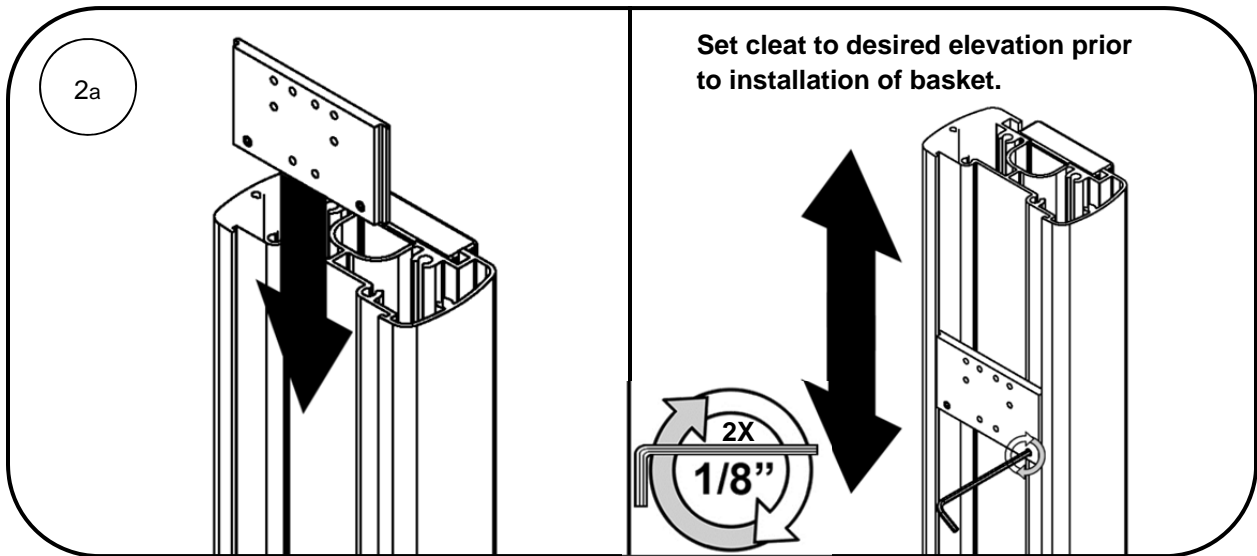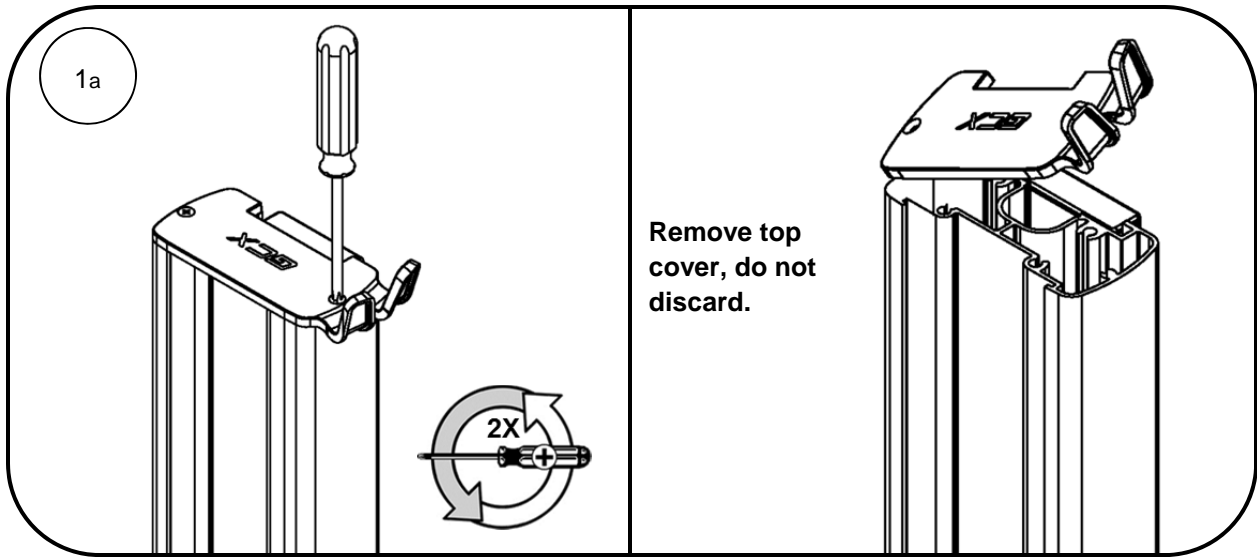


## Roll Stand Configuration Installation

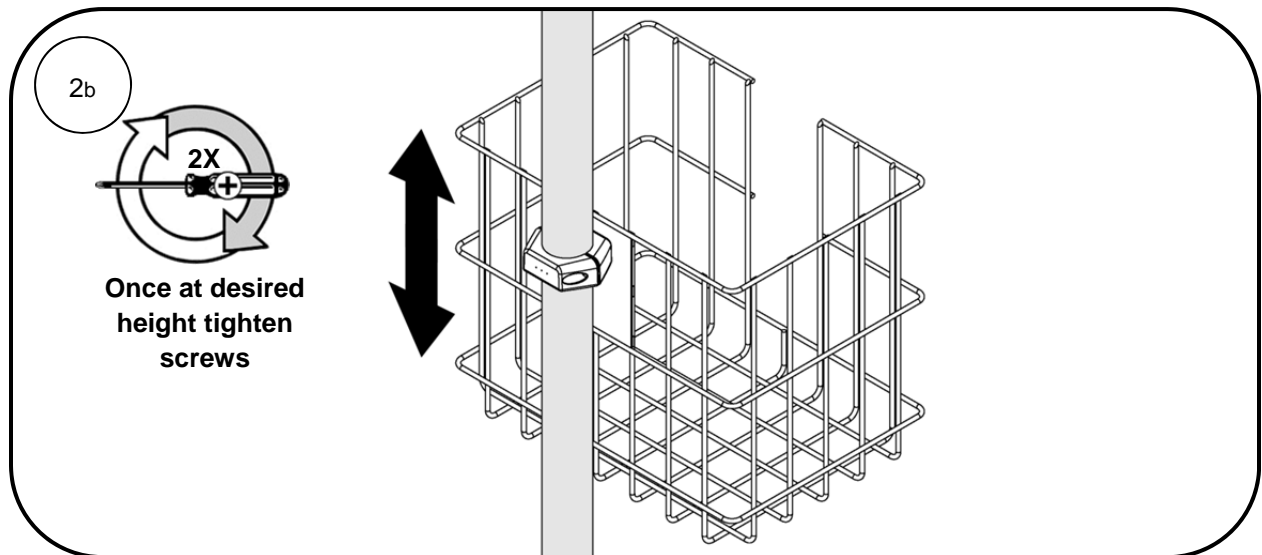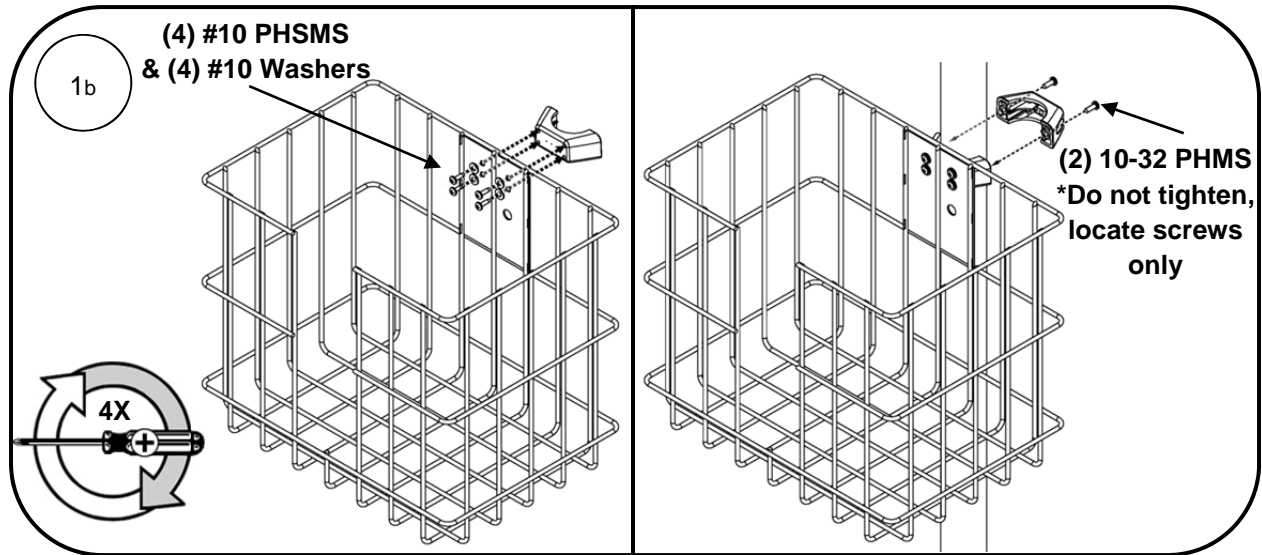

## Channel Configuration Installation

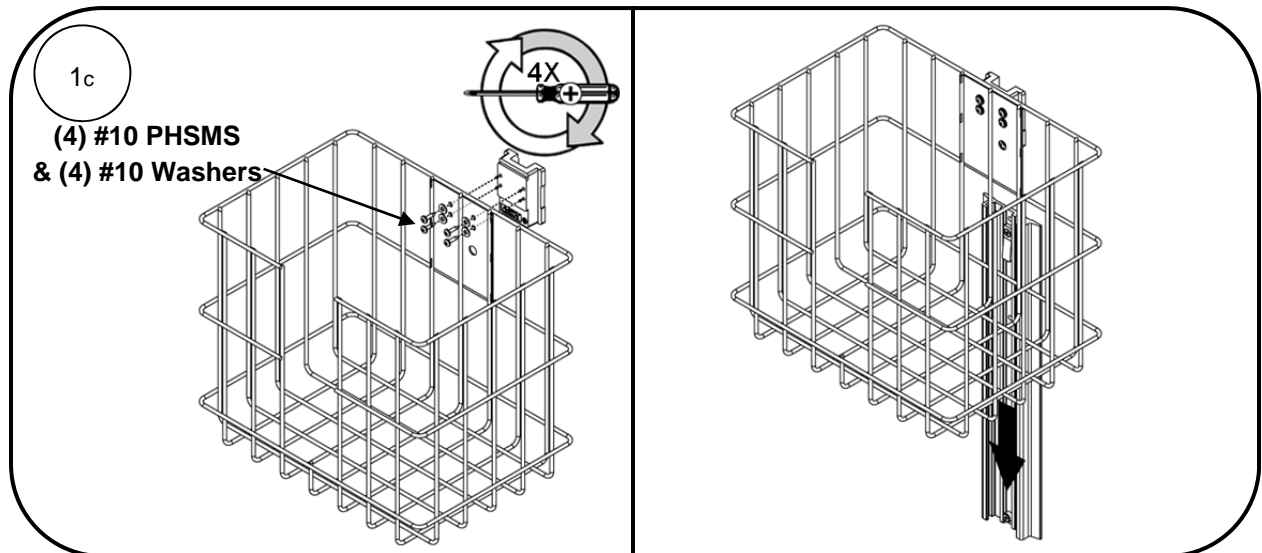

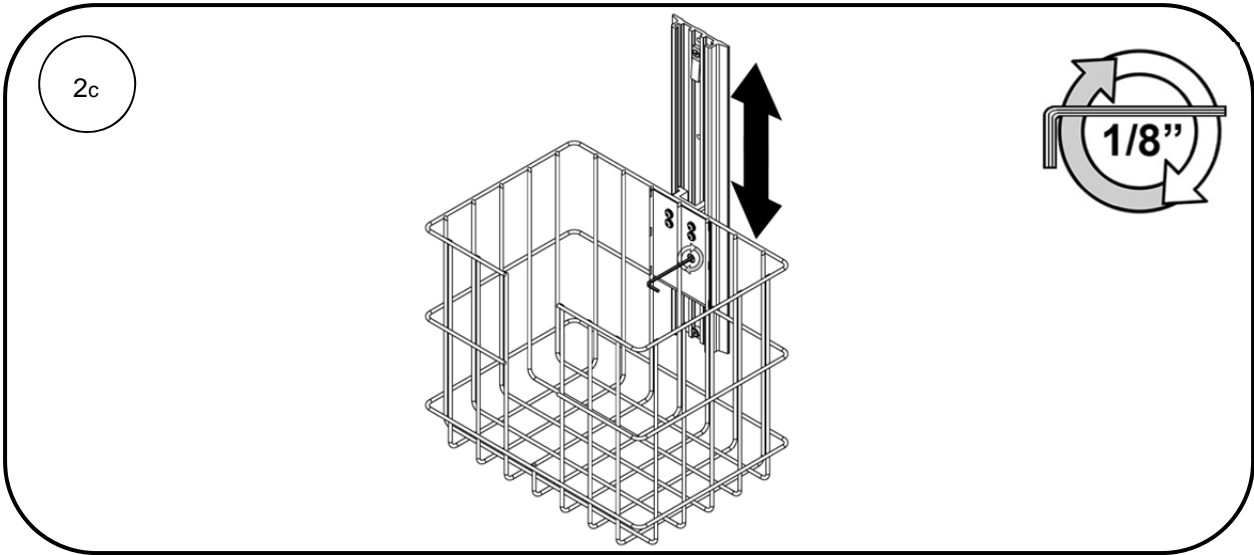

### **Routine Maintenance**

Periodically inspect all fasteners associated with the mounting assembly. Tighten or Adjust as necessary for optimal operation and safety.

### **Cleaning the Mounting Assembly**

1. The mounting assembly may be cleaned with most mild, non-abrasive solutions commonly used in the hospital environment (e.g. diluted bleach, ammonia, or alcohol solutions).
2. The surface finish will be permanently damaged by strong chemicals and solvents such as acetone or trichloroethylene.
3. Steel wool or other abrasive material should never be used.
4. Damaged caused by the use of unapproved substances or processes will not be warranted. We recommend testing of any cleaning solution on a small area of the mounting assembly that is not visible to verify compatibility.
5. Never submerge or allow liquids to enter the mounting assembly. Wipe any cleaning agents off the mounting assembly immediately using a water-dampened cloth. Dry the assembly thoroughly after cleaning.

**CAUTION:** GCX makes no claims regarding the efficacy of the listed chemicals or processes as a means for controlling infection. Consult your hospital's infection control officer or epidemiologist. To clean or sterilize mounted devices or accessory equipment, refer to the specific instructions delivered with those products.