

GCX Mounting Assembly Installation Guide

Handle for 2" Diameter Post

The purpose of this guide is to describe attachment of Handle to post.

Parts Reference

The following parts and hardware are included with this installation kit (see photo for parts, hardware not shown):


Item #	Description	Qty
1	Handle	1
2	#10-32 x 9/16" Pan Head Machine Screw (PHMS)	2



Tools Required: Philips screwdriver.

Attaching Handle to Post

1. Fasten Handle to 2" Post with two (2) #10-32 x 9/16" PHMS. Alternately tighten screws until handle does not rotate on Post.

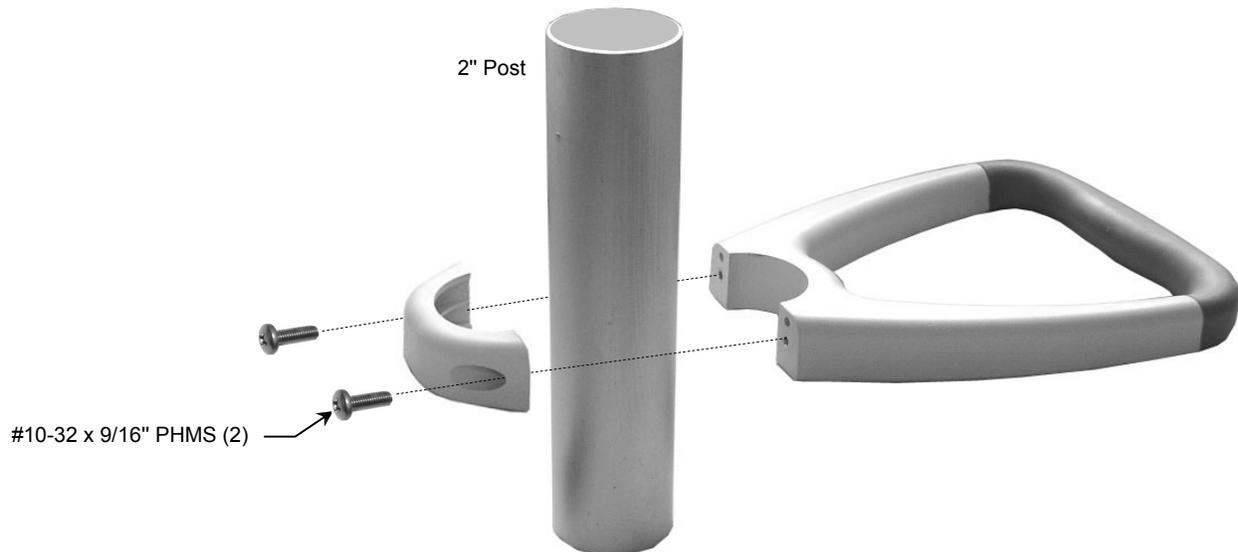

Routine Maintenance

Periodically inspect all hardware associated with the mounting system. Tighten or adjust as necessary for optimal operation and safety.

Cleaning the Mounting Assembly

1. The mounting assembly may be cleaned with most mild, non-abrasive solutions commonly used in the hospital environment (e.g. diluted bleach, ammonia, or alcohol solutions).
2. The surface finish will be permanently damaged by strong chemicals and solvents such as acetone and trichloroethylene.
3. Do not use steel wool or other abrasive material to clean the mounting assembly.
4. Damage caused by the use of unapproved substances or processes will not be covered by warranty. We recommend testing of any cleaning solution on a small area of the arm that is not visible to verify compatibility.
5. Do not submerge or allow liquids to enter the arm. Wipe any cleaning agents off of the arm immediately using a water-dampened cloth. Dry the arm thoroughly after cleaning.

CAUTION: GCX makes no claims regarding the efficacy of the listed chemicals or processes as a means for controlling infection. Consult your hospital's infection control officer or epidemiologist. To clean or sterilize mounted instruments or accessory equipment, refer to the specific instructions delivered with those products.