

GCX Mounting Assembly Installation Guide

Handle for 1.25" or 1" Post




The purpose of this guide is to describe attachment of Handle to 1.25" and 1" Posts.

Parts Reference

The following parts and hardware are included with this installation kit (see photos for parts, hardware not shown):

Item #	Description	Qty
1	Handle	1
2	#10-32 x 9/16" Pan Head Machine Screw (PHMS)	2
3	Handle Spacer for 1" Post	2

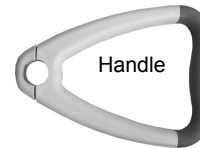


Tools Required: Phillips screwdriver.

Attaching Handle to 1.25" Post

Installation Note: Spacers are not required for a 1.25" post.

1. Fasten Handle to Post with two (2) #10-32 x 9/16" PHMS.

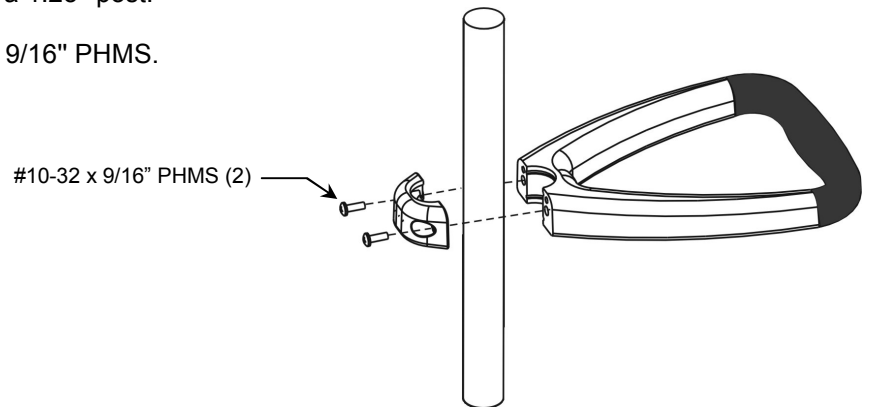

Attaching Handle to 1.0" Post

1. Press one (1) Handle Spacer into the Clamping Block and one (1) Handle Spacer into Handle (below left).
2. Fasten Handle to Post with two (2) #10-32 x 9/16" PHMS.

