


Installation Guide

Dual Power Supply Holster with Power Strip for 2"/5.1cm diameter post

Purpose



- To describe the installation of the Dual Power Supply Holster to a 2" Roll Stand Post.

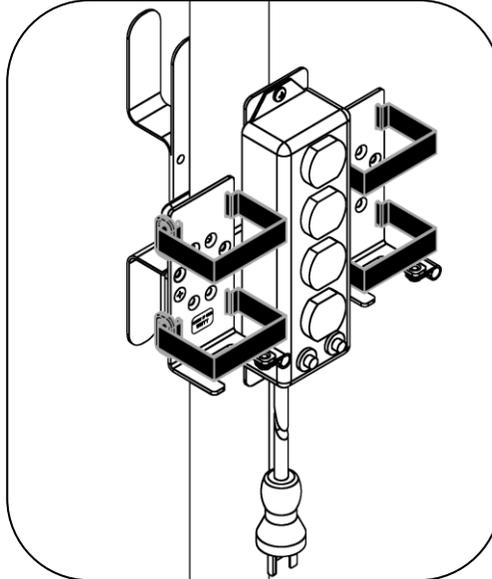

Parts Reference

The following parts and hardware are included with this installation kit (hardware not shown)

Item #	Description	Qty.
1	Dual Power Supply Holster including Power Strip with 12" Velcro Straps, attached to a 2" RS Block with inserts	1
2	Cord Wrap attached to a 2" RS Block w/ Bores	1
3	10-32 X 9/16" PHMS	2
4	5/16 Nylon HD Cable Clamp	2
5	8-32 x 1/4" PHMS	2
6	#8 Flat Washer (.03 THK X .375 OD)	2
7	18" Hook and loop straps	4




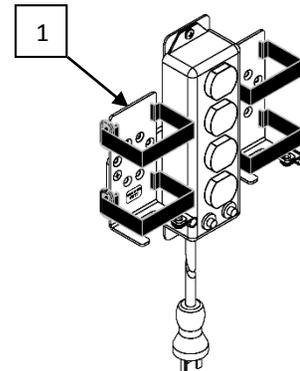
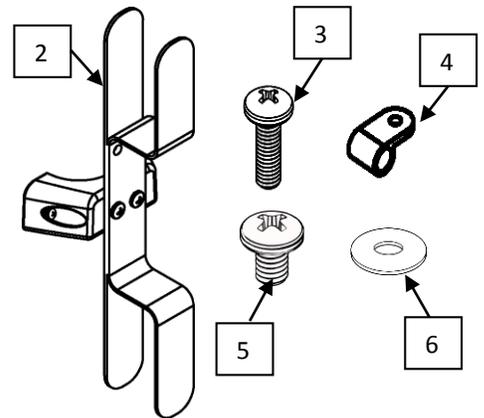


Tools Required: (Not Provided)

- Philips screwdriver

Installing Power Strip Holster and Cord Wrap to Roll Stand Post

Min. Power Supply Size Using 12" [30.5 cm] Hook & Loop Straps	Max. Power Supply Size Using 18" [45.7cm] Hook & Loop Straps
1.4" [3.56 cm] x 2.3" [5.84 cm] or 7.4" [18.8 cm] perimeter	4.0" [10.16 cm] x 4.0" [10.16 cm] or 16" [40.6 cm] perimeter

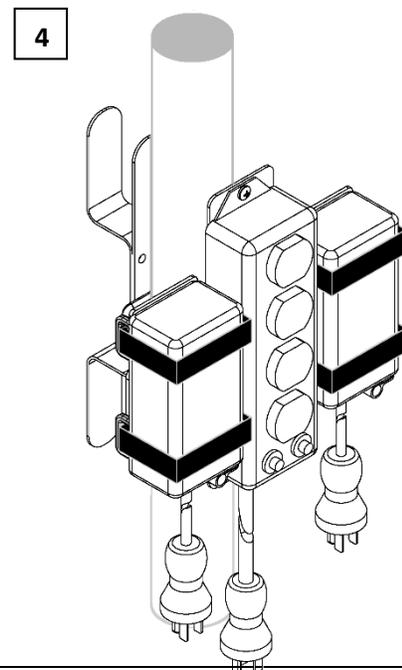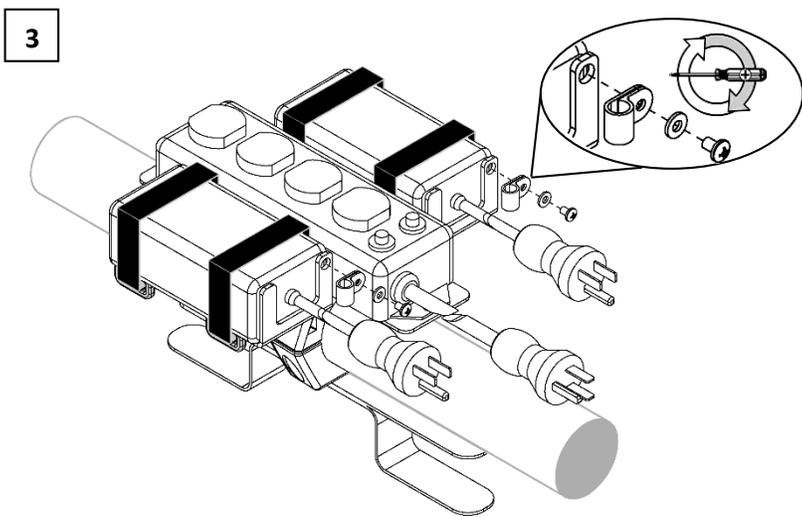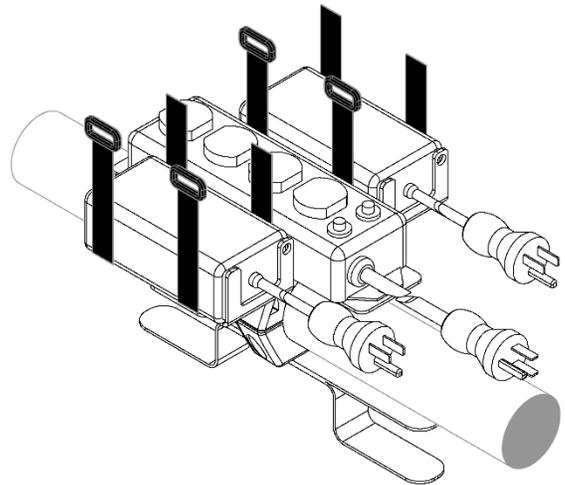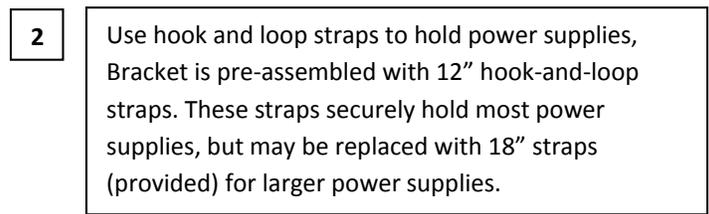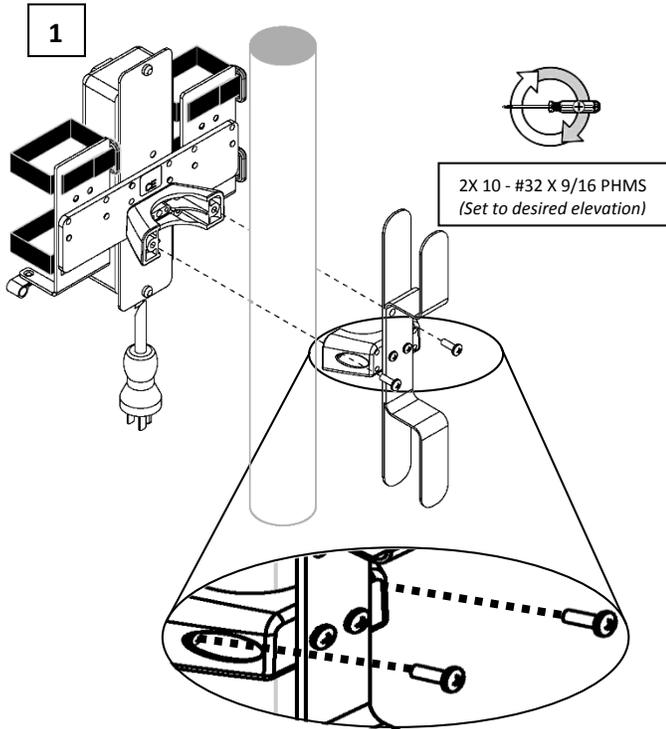

Routine Maintenance

Periodically inspect all fasteners associated with the mounting assembly. Tighten or adjust as necessary for optimal operation and safety.

Cleaning the Mounting Assembly

1. The mounting assembly may be cleaned with most mild, non-abrasive solutions commonly used in the hospital environment (e.g. diluted bleach, ammonia, or alcohol solutions).
2. The surface finish will be permanently damaged by strong chemicals and solvents such as acetone or trichloroethylene.
3. Steel wool or other abrasive material should never be used.
4. Damage caused by the use of unapproved substances or processes will not be warranted. We recommend testing of any cleaning solution on a small area of the mounting assembly that is not visible to verify compatibility.
5. Never submerge or allow liquids to enter the mounting assembly. Wipe any cleaning agents off the arm immediately using a water-dampened cloth. Dry the mounting assembly thoroughly after cleaning.

CAUTION: GCX makes no claims regarding the efficacy of the listed chemicals or processes as a means for controlling infection. Consult your hospital's infection control officer or epidemiologist. To clean or sterilize mounted devices or accessory equipment, refer to the specific instructions delivered with those products