

Installation Guide

Mounting Adapter with 10" Post (1" Dia.) for 5" Slide-on Mounting Plate

The purpose of this guide is to describe mounting of Mounting Adapter on Mounting Plate.

Installation Note: This Mounting Adapter is for instruments or accessories that may be clamped to a 1" diameter, 10" long post.

Parts Reference

The following parts and hardware will be used in this procedure (see photo for part):

Item #	Description	Qty
1	Mounting Adapter with 10" Post	1
2	Thumb Screw, #10-32 x 5/8" Nylon	2



Tools Required: None.

Mounting the Mounting Adapter on Slide-on Mounting Plate

- Slide Mounting Adapter onto Mounting Plate until Adapter contacts Plunger at front of Mounting Plate (below left). Pull Plunger and continue sliding Adapter until Plunger snaps into clearance hole at front of Adapter.
- Mount instrument according to manufacturer's instructions. *Optional:* For additional tightening of mounted instrument, thread Nylon screws into bottom of Base (below right). Tighten screws until they make contact with bottom of instrument. *Note:* Nylon screws must be removed prior to dismounting the Mounting Adapter from the Mounting Plate.

