

## GCX Mounting Assembly Installation Guide

### Variable Height Roll Stand with Storage Bay





The purpose of these instructions is to:



1. Describe attachment of Storage Bay to Roll Stand.
2. Describe attachment of optional Handle to Storage Bay.
3. Describe operation of optional Sliding Laptop Tray.

### Attachment of Storage Bay to Roll Stand

1. Line up the three (3) hole pattern on the Roll Stand with the three (3) hole pattern on the Laptop Tray Bracket and secure using the supplied #10-32 x 9/16" FHMS (3).

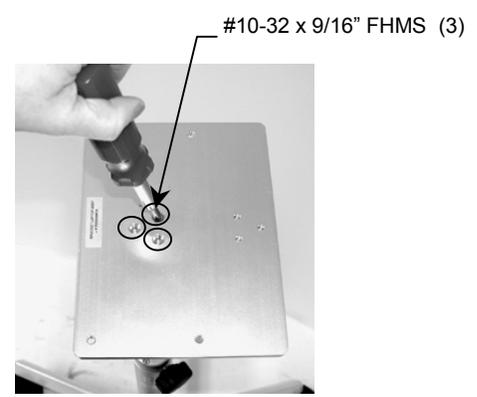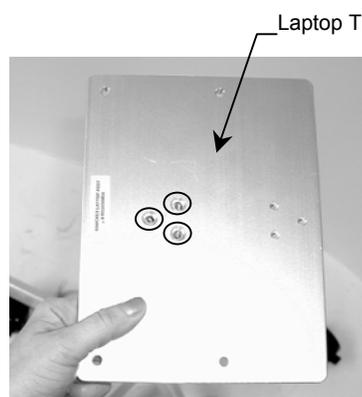

2. Insert the four (4) studs on the bottom of the Storage Bay into the four (4) holes on the Laptop Tray Bracket and secure using the supplied #10-32 Esna Nuts (4).

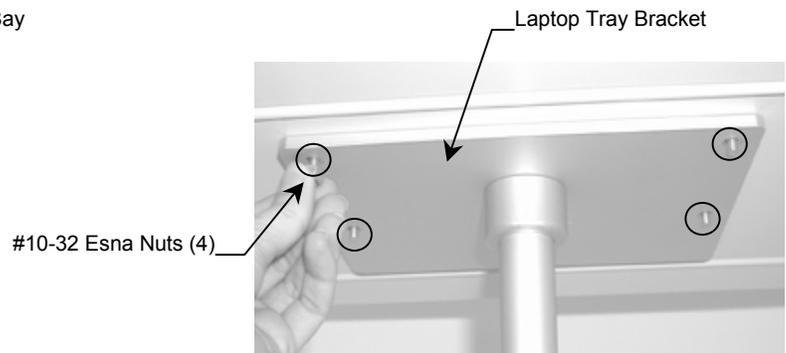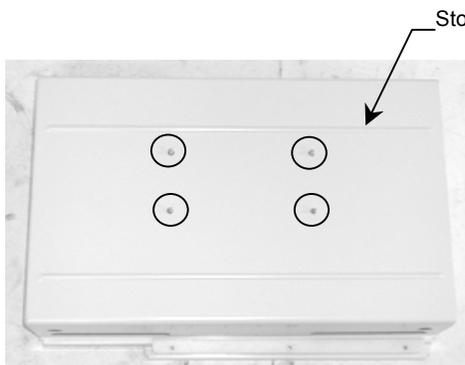


## **Attachment of Handle**

1. Line up the three (3) hole pattern on the Laptop Tray Bracket with the three (3) hole pattern on the Handle, and secure using the supplied #10-32 x 3/8" PHMS (3).



## **Operation of Sliding Laptop Tray**

1. To adjust the position of the Laptop Tray, grasp one side of the tray and slide to the desired location.

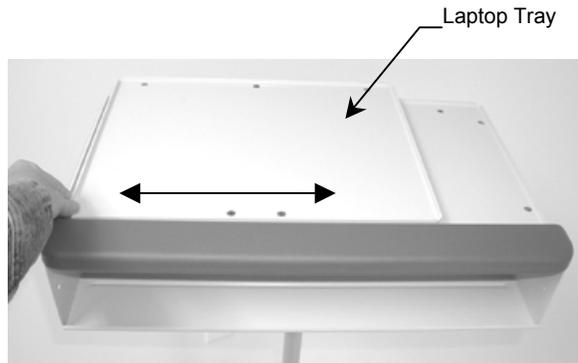

2. Please refer to DU-RS-0006 for Roll Stand Operation and Maintenance.