


Installation Guide

Stor-Locx Adapter Kit-VHRS Applications

Parts Reference

The following parts and hardware are included in this adapter kit.


Item #	Description	Qty
1	Stor-Locx Adapter Bracket	1
2	1/4-20 x 1" Flat Head Socket Cap Screw (FHSCS)	4
3	10-32 x 5/8" Socket Head Cap Screw (SHCS)	3
4	5/32 Hex Wrench	1
5	Screw Head Covers (not shown)	4

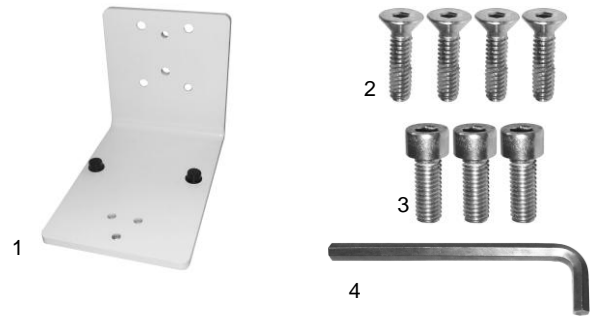


Tools Required: 5/32 Hex Wrench (provided)

Attaching Adapter Bracket to VHRS


- Using a 5/32" hex wrench, attach the Adapter Bracket to the top of the VHRS with (3) 10-32 x 5/8" SHCS.

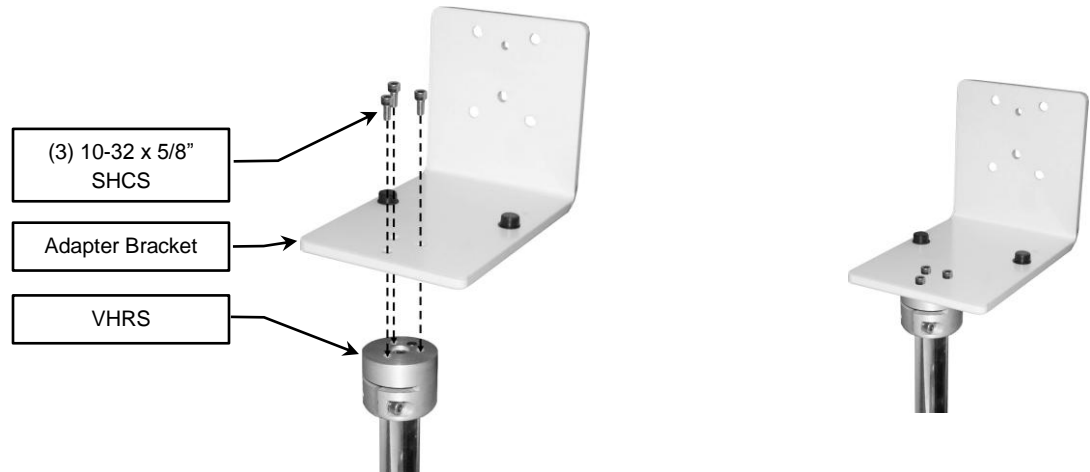


- Remove and discard the (4) mounting screws and Channel Slide attached to the back of the Enclosure.

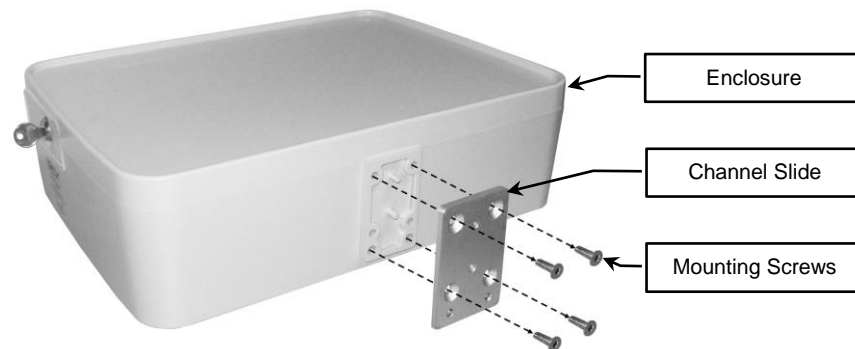

- Align the (2) Guide Pins on the back of the Enclosure with the (2) corresponding Guide Pin holes on the Adapter Bracket as shown (below left). Using a 5/32" Hex Wrench, secure Enclosure to Adapter Bracket with (4) 1/4-20 x 1" FHSCS

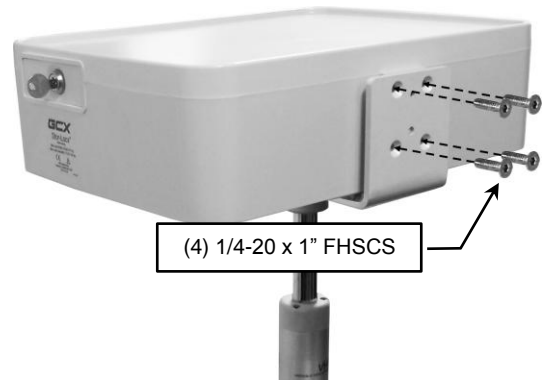

- Apply the (4) self-adhesive Screw Head Covers.

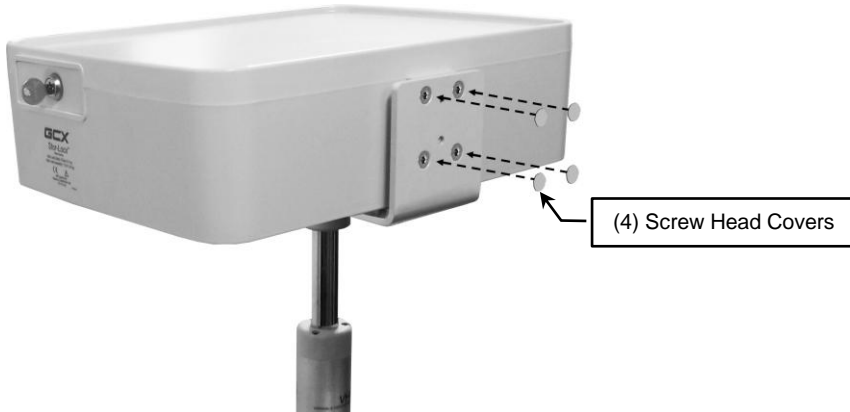

Periodic Maintenance

All fasteners associated with the mounting system should be inspected periodically and tightened as necessary.

Cleaning the Mounting Assembly

- The mounting assembly may be cleaned with most mild, non-abrasive solutions commonly used in the hospital environment (e.g. diluted bleach, ammonia, or alcohol solutions).
- The surface finish will be permanently damaged by strong chemicals and solvents such as acetone or trichloroethylene.
- Steel wool or other abrasive material should never be used.
- Damage caused by the use of unapproved substances or processes will not be warranted. It is recommended that you test any cleaning solution on a small area of the mounting assembly that is not visible to verify compatibility.
- Never submerge or allow liquids to enter the mounting assembly. Wipe any cleaning agents off the assembly immediately, using a water dampened cloth. Dry the assembly thoroughly after cleaning.

CAUTION: GCX makes no claims regarding the efficacy of the listed chemicals or processes as a means for controlling infection. Consult your hospital's infection control officer or epidemiologist. To clean or sterilize mounted devices or accessory equipment, refer to the specific instructions delivered with those products.