

Installation Guide


Mounting Adapter Kit for Mounting Spacelabs Module Housing on Slide-on Mounting Plate

The purpose of this guide is to describe attachment of the Mounting Adapter to the module housing.

Parts Reference

The following parts and hardware are included with this installation kit (major components shown):

Item #	Description	Qty
1	Subplate	1
2	#8-32 x 5/8" Pan Head Machine Screw (PHMS)	4
3	#8 x .030" Flat Washer	4
4	Mounting Adapter	1
5	#8-32 x 5/16" PHMS	4

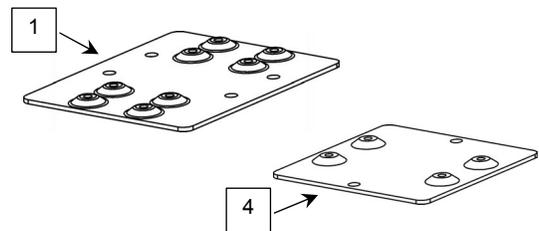


Tools Required


Phillips screwdriver (not provided).

Attaching Mounting Adapter to Module Housing

1. Unscrew feet from bottom of module housing. These mounting holes will be used for attachment of the Subplate.

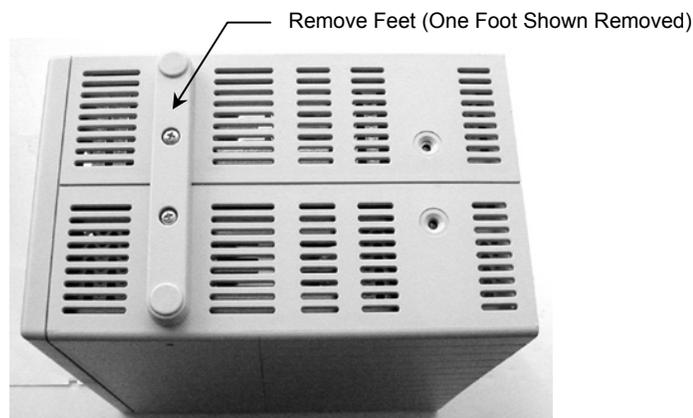

2. Fasten Subplate to bottom of module housing with four (4) #8-32 x 5/8" PHMS and #8 flat washers (below left).
3. Fasten Mounting Adapter to Subplate with four (4) #8-32 x 5/16" PHMS as shown below right.

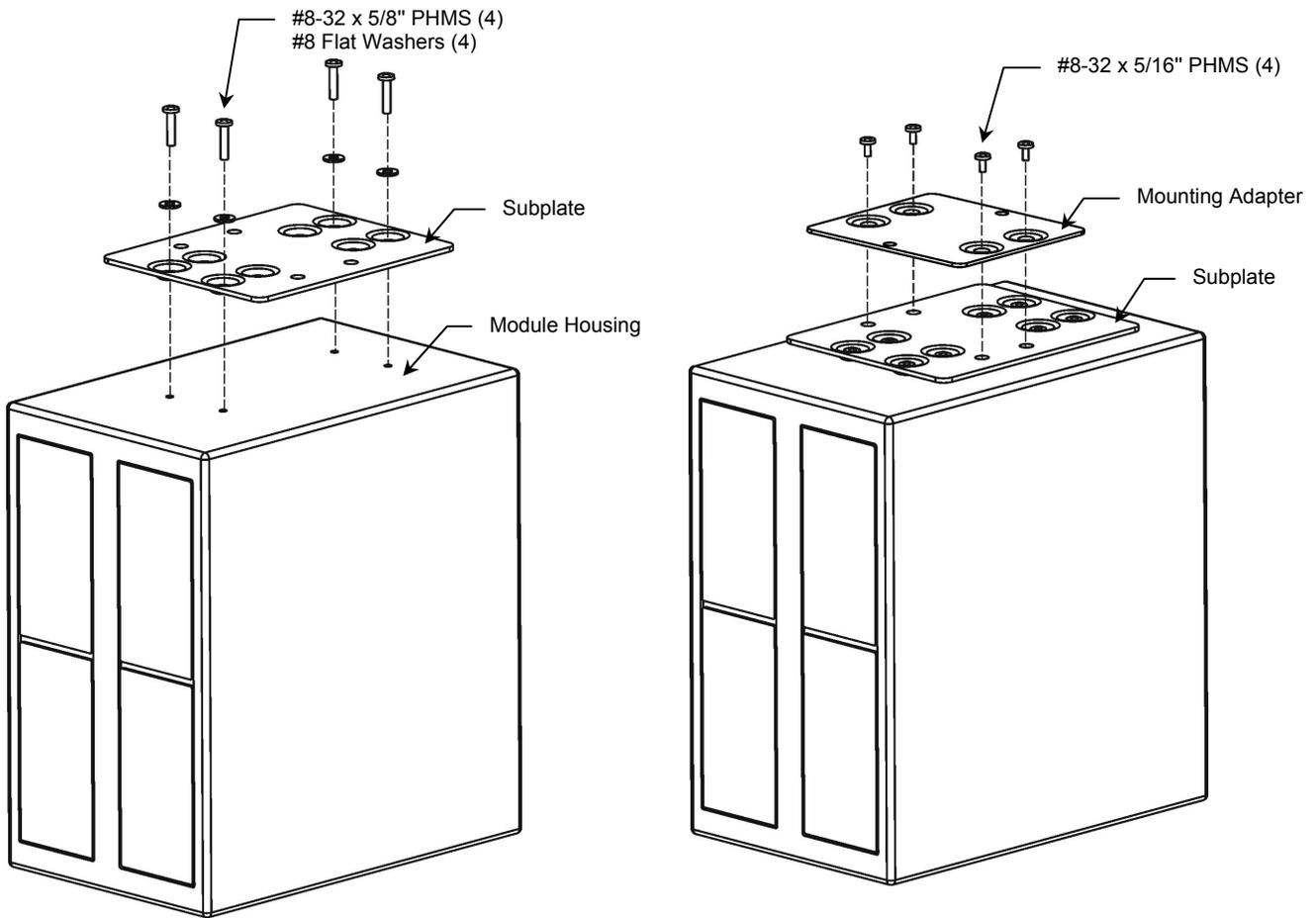

4. Slide module housing laterally into Mounting Plate until Mounting Adapter contacts plunger (below center). Pull plunger and continue sliding housing until plunger snaps into clearance hole at front of Mounting Adapter (right).

