
PolyMount®

**WARNING: REFER TO QUALIFIED PERSONNEL.
FAILURE TO FOLLOW THESE INSTRUCTIONS
MAY RESULT IN SERIOUS INJURY.**

WALL CHANNEL INSTALLATION INSTRUCTIONS FOR NON-SEISMIC SENSITIVE APPLICATIONS

THE MAXIMUM LOAD RATING FOR THIS SYSTEM IS 80 lbs. / 36 Kg
(VB SERIES PRODUCTS) OR 110 lbs. / 50 Kg (FT SERIES PRODUCTS)

WALL CHANNEL ILLUSTRATION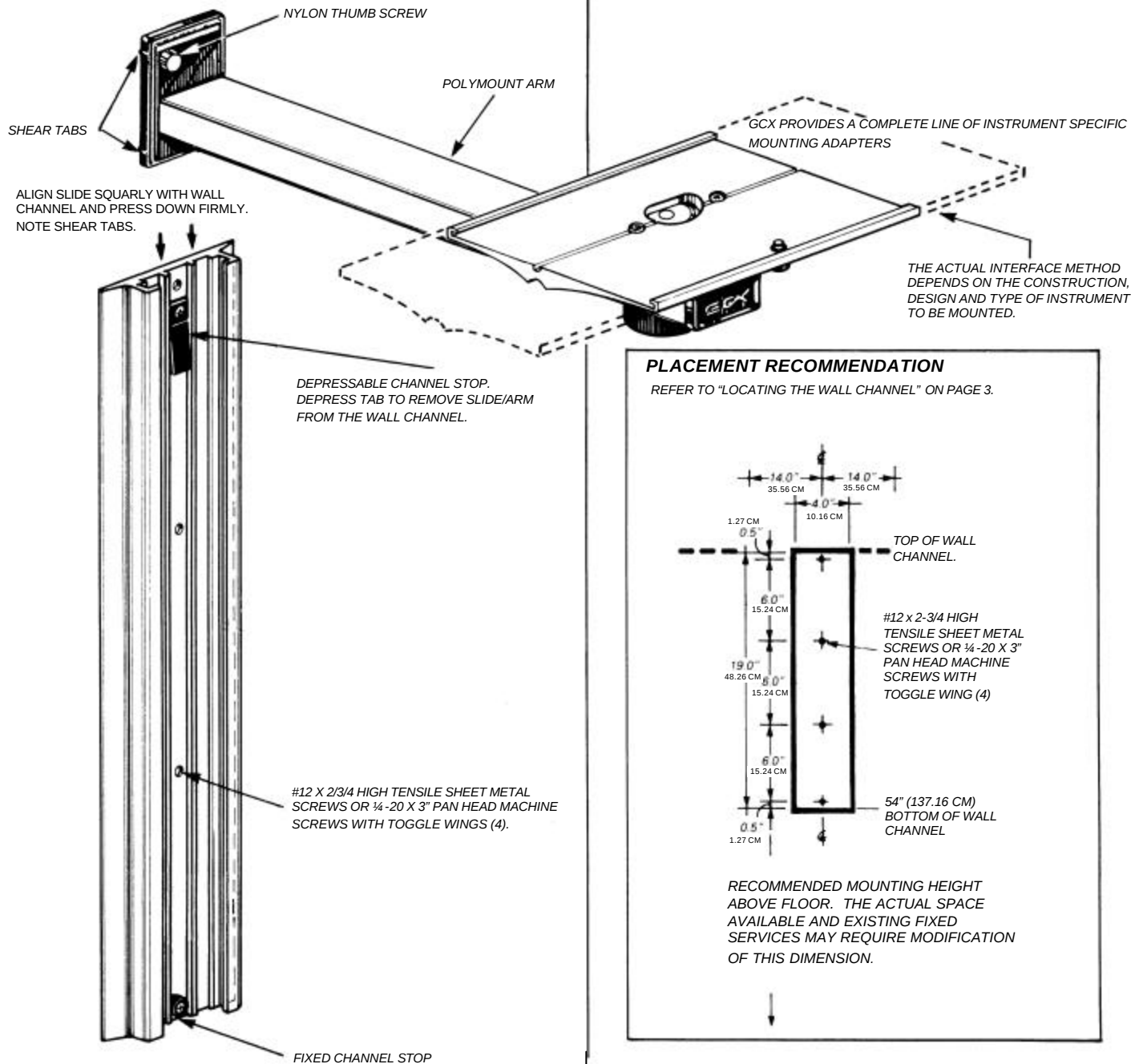


PolyMount®

SUPPORT BACKING FOR 5/8 (15.87 mm) GYPSUM BOARD WALLS FOR WALL MOUNTED INSTRUMENTATION.

THE MAXIMUM LOAD RATING FOR THIS SYSTEM IS 80 lbs. / 36 Kg (VB SERIES PRODUCTS) OR 110 lbs. / 50 Kg (FT SERIES PRODUCTS)

THIS ANCHORAGE WILL ALLOW FOR A QUICK AND SIMPLE INSTALLATION OF THE WALL CHANNEL. 16 GAUGE STEEL WILL PREVENT THE MATERIAL FROM EXTRUDING UP THE SHANK OF THE SCREW (STRIPPING). 11/64" (4.36 mm) DIAMETER HOLES ARE REQUIRED FOR #12 X 2-3/4" HIGH TENSILE, PHILIPS HEAD SCREWS (4) INSTALLED DOWN THE CENTERLINE. (NON-SEISMIC APPLICATIONS).

A SECOND TRACK CAN BE USED FOR BACK TO BACK INSTALLATIONS. A 2" X 4" (5.08 x 10.16 CM) WOOD STUD MAY BE USED IF ALLOWED BY LOCAL BUILDING CODES.

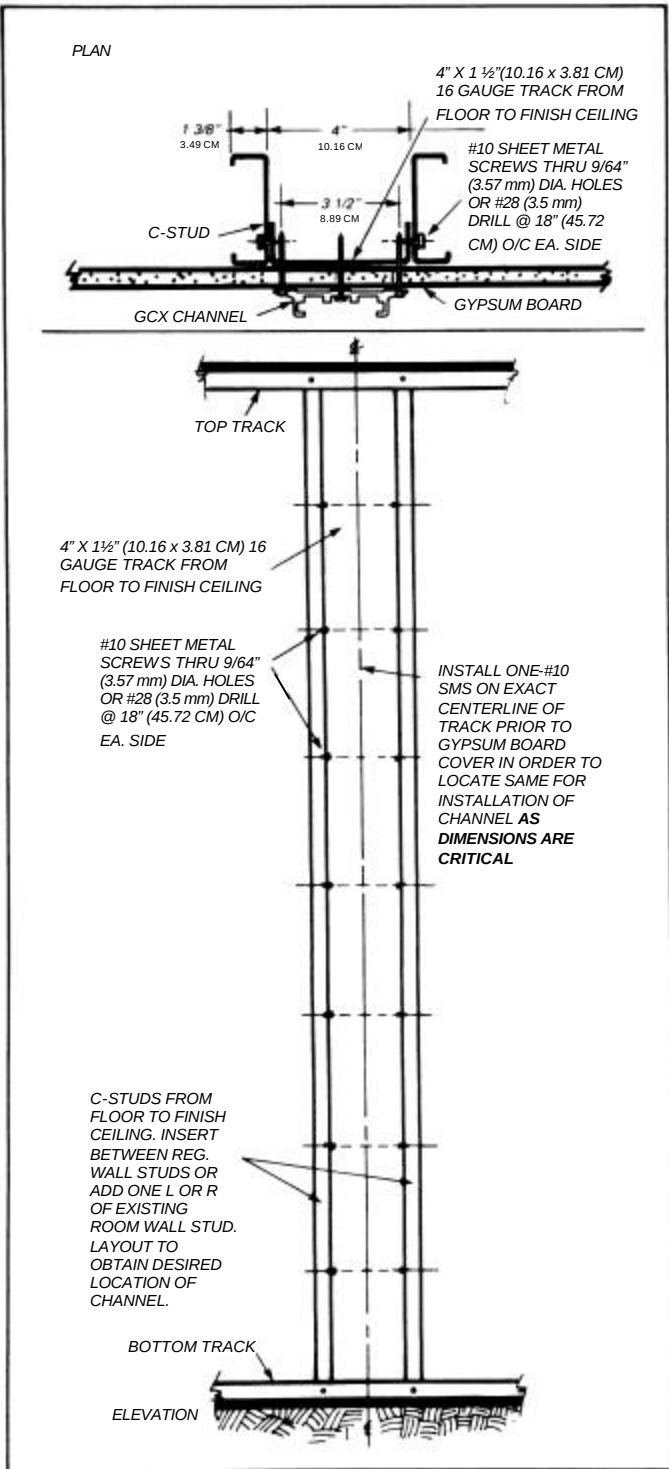MINIMUM REQUIREMENT

16 GAUGE STEEL STUD

2" X 4" (5.08 x 10.16 CM) WOOD STUD

A LATERAL FORCE IN EXCESS OF 80 LBS. (36 Kg) APPLIED 17" (43.18 CM) OUT FROM THE WALL WILL CRACK 5/8" (15.87 mm) THICK GYPSUM BOARD ALONG THE CHANNEL FLANGE. *THIS IS NOT ACCEPTABLE IN SEISMIC SENSITIVE AREAS AND NOT ALLOWED IN CALIFORNIA. SEISMIC WALL CHANNEL INSTALLATION INSTRUCTIONS ARE PRINTED SEPARATELY.*

PolyMount[®]

THE MAXIMUM LOAD RATING FOR THIS SYSTEM IS 80 lbs. / 36 Kg
(VB SERIES PRODUCTS) OR 110 lbs. / 50 Kg (FT SERIES PRODUCTS)

INSTRUCTIONS AND THE ILLUSTRATIONS COVERING THE SPECIFIC INSTRUMENT TO BE MOUNTED SHOULD BE REVIEWED PRIOR TO INSTALLATION OF THE WALL CHANNEL.

It shall be the responsibility of the hospital, its consultants and/or contractors to determine that the wall is adequate to safely mount instrumentation. This includes the selection of appropriate fasteners and the proper installation of the same. In new construction and remodeling work where the wall covering can be entirely or partly removed, a 16 gauge steel stud should be located for the purpose of attaching the wall channel. The following is provided as guideline information only.

LOCATING THE WALL CHANNEL:

Generally, the bottom edge of a 19 inch (48.26 CM) Wall Channel is placed 56 inches (142.24 CM) above the floor, plus or minus 4 inches. Devices may require lower channel placement. The bottom of the device will be 4-1/2 inches (10.16 CM) above the bottom of the channel (6-1/2 inches (16.51 CM) when using 15 inch (38.10 CM) support arm with a diagonal brace) with the Arm at its lowest position. Access to device controls should be considered before mounting the wall channel.

Allow clearance on either side of the channel centerline to clear objects such as over-bed lighting, privacy curtains, adjacent walls or columns, door swing arcs, etc. Power and signal outlets should be considered when selecting a channel location. Avoid oxygen, vacuum and air outlets and space for attendant flow meters and regulators. Do not place any portion of the device over a patient bed.

DRYWALL (GYPSUM BOARD) OVER 16 GAUGE SHEET METAL OR WOOD STUDS:

The Wall Channel must be anchored into the stud.

Locate the exact centerline of the stud. Drive a #4 finishing nail through the drywall to contact the stud. Withdraw and drive again 3/8" (9.52 mm) right and left until the edges of the stud have been located and hence the centerline determined. The wall channel's 4 inch (10.16 CM) width will cover these exploratory holes.

Drill 11/64" (17 mm) diameter holes for #12 X 2-3/4" high tensile Philips head screws in a single 16 gauge stud through channel centerline, (4 places). Install the top screw and level the channel, then mark and drill the other holes.

DRYWALL (GYPSUM BOARD) OVER SHEET METAL STUDS LESS THAN 16 GAUGE:

The Wall Channel must be anchored into the stud.

Locate the centerline of the stud per the above instructions. Use 1/4-20 X 3" pan head machine screws and toggle wings. Drill 11/16" diameter holes with a sharp speed bore or twist drill (requires pilot hole). Note that the round stop at the bottom of the channel goes over the shank of the bottom screw. Insert screws through Wall Channel holes then affix the toggle wings to the screws. Insert toggle wings into previously drilled holes. Use a level to make sure the channel is vertical prior to final seating of the screws.

PolyMount®

THE MAXIMUM LOAD RATING FOR THIS SYSTEM IS 80 lbs. / 36 Kg
(VB SERIES PRODUCTS) OR 110 lbs. / 50 Kg (FT SERIES PRODUCTS)

PLASTER COAT OVER EXPANDED METAL LATH ON STEEL STUDS, HOLLOW TILE, HOLLOWBLOCK:

Drill 5/8" (16 mm) holes. Broach the holes in hard materials to allow for the toggle wing to pass through. Insert screws through Wall Channel holes then affix the toggle wings to the screws. Insert toggle wings into previously drilled holes. Use a level to make sure the channel is vertical prior to final seating of the screws.

CONCRETE WALLS:

REFER TO QUALIFIED PERSONNEL. Contact GCX for concrete installation addendum.

THROUGH WALL, BACK TO BACK INSTALLATIONS:

Sufficient length 1/4-20 machine screws, nuts, washers and/or back plates must be obtained to mount the channel by drilling all the way through the thin partition wall. If two channels are to be mounted back to back, internally threaded sleeves, exact length screws and specific procedures are available from GCX upon request. A Portalalign drill fixture is recommended for this procedure.

SPECIAL APPLICATIONS:

Channels can be attached to a GCX 4" X 4" (10.16 x 10.16 CM), 1/8" (3.17 mm) wall aluminum support column or certain mullions, etc.. Contact GCX to discuss solutions to wall mounting problems for any situation not covered by these guidelines.

ALTHOUGH CONSIDERABLE EFFORT HAS BEEN MADE TO ENSURE THE SAFETY OF THE ABOVE INSTALLATION AND/OR GUIDELINES, THE INSTALLATION ITSELF IS BEYOND THE CONTROL OF GCX CORPORATION. ACCORDINGLY, GCX CORPORATION WILL NOT BE RESPONSIBLE FOR THE FAILURE OF ANY SUCH INSTALLATION.

FOR ADDITIONAL INFORMATION, PLEASE CONTACT:

GCX Corporation
3875 Cypress Drive
Petaluma, CA 94954-5635
707.773.1100
800.228.2555 US/CAN
Fax: 707.773.1180
E-mail: sales@gcx.com
Web: www.gcx.com

GCX, POLYMOUNT, POLYQUIP AND POWERMOUNT ARE REGISTERED TRADEMARKS OF GCX CORPORATION.

CONFIDENTIAL MATERIAL

© 1987 GCX CORPORATION ALL RIGHTS RESERVED

THIS MATERIAL IS THE PROPRIETARY INFORMATION OF GCX CORPORATION AND ANY USE, REPRODUCTION, OR DISTRIBUTION OF THE MATERIAL CONTAINED HEREIN, UNLESS SPECIFICALLY AUTHORIZED BY GCX CORPORATION IS EXPRESSLY FORBIDDEN AND WOULD VIOLATE THE RIGHTS AND INTERESTS OF GCX CORPORATION.

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