

Lilitab H Pro Head Unit

Assembly Instructions

v7.0



lilitab

39 Larkspur Street | Suite B
San Rafael, CA 94901

t: 888.705.0190
e: support@lilitab.com

www.lilitab.com

TABLE OF CONTENTS

Packing List.....	2
Assembly Instructions	3
Cleaning Instructions.....	4



← Scan to watch
assembly video on
our YouTube Channel!

GOT QUESTIONS? WE'RE HERE TO HELP!

EMAIL support@lilitab.com **PHONE** 888.705.0190

PACKING LIST



1 x Head Unit and Faceplate

ASSEMBLY INSTRUCTIONS



1. Remove faceplate:

A. On the back of the Head Unit, pinch together the faceplate release clips.




3. **Install faceplate** by sliding the faceplate onto the Head Unit in the same fashion you removed it, but slide right, towards the home button.

NOTE: The faceplate only fits in one direction—therefore if it doesn't slide easily, please reverse the direction—do not force it.

Simply reverse these steps to remove your tablet.

Refer to your Lilitab Mount owner's manual for instructions on attaching the Head Unit.

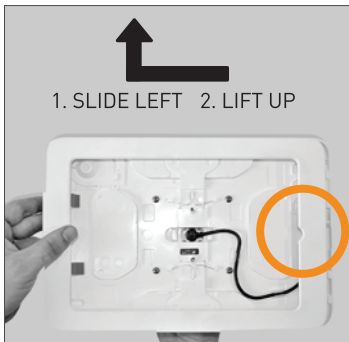

B. While still pinching the clips, flip the Head Unit over and slide the faceplate left, away from the home button, and lift up to remove the faceplate.



2. **Insert your tablet.** Lay the Head Unit face up and connect the charge cable.

NOTE: Before inserting your tablet, ensure the charge port is on the same side as the charge cable.

CLEANING INSTRUCTIONS



Cleaning kit available on our website in the accessories section.

1. **Spray the Brilliantize** onto one of the cloths provided.

WARNING: Other cloth/material may scratch the surface of your Lilitab.



2. DO NOT spray directly onto your Lilitab!



3. **Rub gently** in a circular motion to remove fingerprints and dirt.

WARNING: Applying excess pressure can scratch the surface of your Lilitab.